



**HBV Longitudinal Panel  
Part Number: HBV6501**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

**\*Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK
6501-01	03/22/01	0.06	36.4	>40.8	5390	0.04	0.13	>6.9
6501-02	04/05/01	0.06	35.1	>40.8	7470	0.03	0.12	>6.9
6501-03	04/19/01	0.06	34.6	>40.8	5530	0.06	0.04	4.8
6501-04	05/03/01	0.06	34.6	>40.8	2260	0.07	0.03	3.6
6501-05	05/17/01	0.07	35.9	>40.8	519	0.07	0.03	2.6
6501-06	05/31/01	0.07	35.2	29.1	29.6	0.08	0.03	2.2
6501-07	06/13/01	0.06	34.5	0.4	0.11	0.06	0.02	1.7
6501-08	06/28/01	0.07	34.1	0.04	0.09	0.09	0.02	1.4
6501-09	07/12/01	0.07	33.7	0.2	0.08	0.09	0.03	1.1
6501-10	07/25/01	0.06	33.6	0.04	0.09	0.03	0.02	1.2
6501-11	08/09/01	0.05	31.9	0.09	0.08	0.03	0.02	1.12
6501-12	08/22/01	0.09	31.2	0.09	0.11	0.04	0.03	1.12

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: UNK
6501-01	3.45	0.4	2.86	0.3	2.0	0.14	0.2
6501-02	3.00	0.4	1.41	0.2	2.8	0.13	0.2
6501-03	2.33	0.5	1.31	0.3	1.6	0.11	0.3
6501-04	1.77	0.4	QNS	0.3	1.4	0.11	0.2
6501-05	1.47	0.4	QNS	0.2	2.1	0.11	0.2
6501-06	QNS	0.5	QNS	0.3	QNS	0.12	0.2
6501-07	1.11	0.5	2.92	0.7	4.2	0.11	0.2
6501-08	0.91	0.6	2.41	0.8	7.0	0.11	0.2
6501-09	0.80	0.8	1.96	0.8	7.9	0.12	0.2
6501-10	0.79	0.3	2.48	1.05	8.7	0.10	0.3
6501-11	0.68	0.4	1.60	1.09	8.4	0.11	0.3
6501-12	0.62	0.3	1.68	1.2	8.1	0.11	0.2

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6501 Rev02  
Effective Date: 09/01/2021