

# ZeptoMetrix®

## HBV Longitudinal Panel Part Number: HBV6504

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

**\*Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK
6504-01	3/26/01	0.09	36.5	>40.8	7840	0.06	0.10	>6.9
6504-02	4/6/01	0.09	37.0	>40.8	6310	0.06	0.16	>6.9
6504-03	4/19/01	0.08	37.2	>40.8	1220	0.06	0.07	4.9
6504-04	5/3/01	0.08	36.8	33.2	67.9	0.08	0.05	3.7
6504-05	5/22/01	0.08	36.4	0.02	0.11	0.08	0.03	2.1
6504-06	6/3/01	0.07	36.7	0.06	0.12	0.08	0.03	2.2
6504-07	6/14/01	0.06	35.5	0.02	0.10	0.08	0.02	1.9
6504-08	6/28/01	0.08	36.8	0.02	0.11	0.06	0.02	1.9
6504-09	7/12/01	0.08	37.4	0.02	0.11	0.06	0.02	1.9
6504-10	7/25/01	0.07	36.4	0.02	0.09	0.04	0.02	1.43
6504-11	8/9/01	0.07	30.7	0.05	0.08	0.04	0.02	1.03
6504-12	8/22/01	0.40	29.1	0.07	0.06	0.03	0.02	0.79

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: UNK
6504-01	3.44	0.9	0.46	0.4	2.6	0.72	0.4
6504-02	3.13	0.9	0.5	0.2	1.9	0.36	0.3
6504-03	2.16	0.6	1.11	0.4	2.0	0.13	0.2
6504-04	1.57	0.5	2.07	0.3	1.8	0.12	0.1
6504-05	0.95	0.4	4.44	0.4	2.0	0.11	0.1
6504-06	0.88	0.4	3.30	0.3	2.3	0.13	0.1
6504-07	0.82	0.4	7.01	0.4	2.1	0.11	0.2
6504-08	0.78	0.4	4.88	0.2	2.6	0.11	0.1
6504-09	0.73	0.3	8.55	0.4	1.7	0.11	0.2
6504-10	0.58	0.1	7.69	0.57	3.7	0.12	0.3
6504-11	0.68	0.5	1.15	0.89	7.0	0.12	0.3
6504-12	0.59	0.4	0.87	0.65	6.9	0.10	0.2

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6504 Rev02  
Effective Date: 09/01/2021