



**HBV Longitudinal Panel  
Part Number: HBV6506**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECI ANTI-HIV ½	ORTHO VITROS ECI ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECI HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECI ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK
6506-01	03/30/01	0.05	0.08	>40.8	4120	0.1	0.04	>6.9
6506-02	04/12/01	0.07	0.09	>40.8	7810	0.05	0.03	>6.9
6506-03	04/26/01	0.06	0.10	5.7	10.2	0.08	0.02	>6.9
6506-04	05/10/01	0.08	0.14	0.02	0.10	0.30	0.02	>6.9
6506-05	05/24/01	0.06	0.06	0.04	0.08	0.08	0.02	>6.9
6506-06	06/07/01	0.06	0.06	0.02	0.09	0.10	0.02	6.1
6506-07	06/21/01	0.05	0.03	0.02	0.08	0.08	0.02	5.2
6506-08	07/05/01	0.06	0.03	0.02	0.08	0.09	0.01	4.7
6506-09	07/19/01	QNS	QNS	0.08	QNS	0.04	QNS	1.14
6506-10	08/02/01	0.05	0.03	0.02	0.08	0.03	0.02	3.9
6506-11	08/18/01	0.46	0.11	0.13	0.10	0.03	0.03	3.6
6506-12	08/30/01	0.22	0.07	0.08	0.09	0.02	0.01	3.3
6506-13	11/27/01	0.10	0.06	0.05	0.08	0.07	0.02	1.74
6506-14	12/25/01	0.09	0.05	0.04	0.08	0.06	0.02	2.1

PANEL NUMBER	ORTHO VITROS ECI ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECI ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECI ANTI-HBs QUANT	ORTHO VITROS ECI HBeAg	DIASORIN HBeAg
	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: UNK	TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: UNK
6506-01	3.88	0.08	12.8	0.4	1.7	0.12	0.2
6506-02	3.86	0.03	34.5	0.4	2.2	0.11	0.2
6506-03	3.70	0.02	27.6	0.4	1.9	0.10	0.1
6506-04	3.34	0.03	QNS	0.5	2.1	0.11	0.1
6506-05	2.87	0.02	27.6	0.5	2.8	0.12	0.3
6506-06	2.37	0.03	26.7	0.8	7.1	0.13	0.1
6506-07	2.13	0.01	37.6	1.1	20.6	0.10	0.2
6506-08	1.73	0.02	34.4	2.0	34.9	0.39	0.2
6506-09	QNS	0.3	QNS	1.15	QNS	QNS	0.01
6506-10	1.59	0.02	23.8	6.3	77.8	0.11	0.01
6506-11	1.25	0.03	29.5	6.5	77.3	0.10	0.03
6506-12	1.15	0.03	QNS	6.6	67.3	0.11	0.01
6506-13	0.85	0.06	17.0	5.2	59.7	0.10	0.01
6506-14	0.73	0.04	20.7	5.7	54.2	0.11	0.03

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)