



**HBV Longitudinal Panel
Part Number: HBV6507**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

***Donor is co-infected with Hepatitis C.**

| PANEL NUMBER | DATE OF DRAW | ORTHO VITROS ECi ANTI-HIV ½ | ORTHO VITROS ECi ANTI-HCV | ABBOTT HBsAg | ORTHO VITROS ECi HBsAg | ABBOTT ANTI-HBc TOTAL | ORTHO VITROS ECi ANTI-HBc TOTAL | ABBOTT ANTI-HBc IgM |
|--------------|--------------|-----------------------------|---------------------------|----------------|------------------------|-----------------------|---------------------------------|---------------------|
| | | TEST DATE: 8/4/2014 | TEST DATE: 8/4/2014 | TEST DATE: N/A | TEST DATE: 8/4/2014 | TEST DATE: N/A | TEST DATE: 8/4/2014 | TEST DATE: N/A |
| 6507-01 | 04/16/01 | 0.06 | 25.5 | >40.8 | 5240 | 0.04 | 0.02 | >6.9 |
| 6507-02 | 04/30/01 | 0.06 | 26.4 | >40.8 | 7400 | 0.06 | 0.05 | >6.9 |
| 6507-03 | 05/14/01 | 0.12 | 19.7 | >40.8 | 2640 | 0.07 | 0.01 | 6.0 |
| 6507-04 | 05/28/01 | 0.12 | 17.2 | >40.8 | 2130 | 0.08 | 0.01 | 5.9 |
| 6507-05 | 06/11/01 | 0.08 | 16.4 | 0.04 | 0.12 | 0.08 | 0.03 | 4.0 |
| 6507-06 | 06/25/01 | 0.07 | 14.1 | 0.06 | 0.13 | 0.09 | 0.04 | 3.9 |
| 6507-07 | 07/09/01 | 0.08 | 11.7 | 0.04 | 0.12 | 0.09 | 0.02 | 3.7 |
| 6507-08 | 07/23/01 | 0.06 | 11.1 | 0.15 | 0.15 | 0.04 | 0.04 | 3.8 |
| 6507-09 | 08/06/01 | QNS | QNS | 0.15 | QNS | 0.03 | QNS | 2.3 |
| 6507-10 | 08/21/01 | 0.08 | 34.8 | 0.14 | 0.12 | 0.03 | 0.03 | 2.7 |
| 6507-11 | 09/03/01 | 0.08 | 35.0 | 0.09 | 0.13 | 0.04 | 0.03 | 2.6 |
| 6507-12 | 09/17/01 | 0.07 | 35.0 | 0.02 | 0.11 | 0.04 | 0.03 | 2.3 |
| 6507-13 | 11/27/01 | 0.06 | 34.6 | 0.04 | 0.13 | 0.08 | 0.02 | 1.6 |
| 6507-14 | 12/11/01 | 0.08 | 34.4 | 0.09 | 0.11 | 0.09 | 0.03 | 1.4 |

| PANEL NUMBER | ORTHO VITROS ECi ANTI-HBc IgM | DIASORIN ANTI-Hbe | ORTHO VITROS ECi ANTI-Hbe | ABBOTT ANTI-HBs QUANT | ORTHO VITROS ECi ANTI-HBs QUANT | ORTHO VITROS ECi HBeAg | DIASORIN HBeAg |
|--------------|-------------------------------|-------------------|---------------------------|-----------------------|---------------------------------|------------------------|----------------|
| | TEST DATE: 8/4/2014 | TEST DATE: N/A | TEST DATE: 8/4/2014 | TEST DATE: N/A | TEST DATE: 8/4/2014 | TEST DATE: 8/4/2014 | TEST DATE: N/A |
| 6507-01 | 3.85 | 0.6 | 0.34 | 0.3 | 4.3 | 4.01 | 0.7 |
| 6507-02 | 3.80 | 0.6 | 0.46 | 0.3 | 3.4 | 0.28 | 0.2 |
| 6507-03 | 2.52 | 0.6 | 1.04 | 0.2 | 3.9 | 0.12 | 0.1 |
| 6507-04 | 2.58 | 0.7 | 0.59 | 0.2 | 4.1 | 0.12 | 0.2 |
| 6507-05 | 2.00 | 0.3 | 3.43 | 0.4 | 4.5 | 0.12 | 0.3 |
| 6507-06 | 1.90 | 0.2 | 10.2 | 0.5 | 8.0 | 0.12 | 0.1 |
| 6507-07 | 1.74 | 0.2 | 15.9 | 0.5 | 13.2 | 0.12 | 0.2 |
| 6507-08 | 1.66 | 0.1 | 16.6 | 1.9 | 28.2 | 0.13 | 0.3 |
| 6507-09 | QNS | 0.3 | QNS | 0.55 | QNS | QNS | 0.3 |
| 6507-10 | 1.45 | 0.1 | 26.3 | 5.3 | 74.2 | 0.18 | 0.4 |
| 6507-11 | 1.39 | 0.1 | 27.2 | 4.8 | 109 | 0.13 | 0.4 |
| 6507-12 | 1.28 | 0.1 | 28.9 | 4.4 | 113 | 0.11 | 0.4 |
| 6507-13 | 0.97 | 0.03 | 27.5 | 17.8 | 374 | 0.12 | 0.3 |
| 6507-14 | 0.89 | 0.04 | 25.7 | 19.0 | 410 | 0.12 | 0.3 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6507 Rev03
Effective Date: 09/01/2021