

ZeptoMetrix®

HBV Longitudinal Panel Part Number: HBV6510

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 8/8/2014	ORTHO VITROS ECi ANTI-HCV TEST DATE: 8/8/2014	ABBOTT HBsAg TEST DATE: N/A	ORTHO VITROS ECi HBsAg TEST DATE: 8/8/2014	ABBOTT ANTI-HBc TOTAL TEST DATE: N/A	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/8/2014	ABBOTT ANTI-HBc IgM TEST DATE: N/A
6510-01	5/22/01	0.07	0.07	34.7	986	0.03	0.07	>6.9
6510-02	6/5/01	0.07	0.09	>40.8	1850	0.03	0.04	>6.9
6510-03	6/19/01	0.08	0.05	>40.8	5310	0.04	0.02	4.8
6510-04	7/3/01	0.08	0.04	>40.8	7510	0.04	0.04	4.1
6510-05	7/17/01	0.08	0.05	>38	7320	0.03	0.02	2.7
6510-06	7/31/01	0.08	0.08	>38	1270	0.04	0.25	5.5
6510-07	8/14/01	0.08	0.06	0.08	0.05	0.04	0.10	2.9
6510-08	8/28/01	0.07	0.05	0.23	0.12	0.04	0.08	2.8
6510-09	9/11/01	0.08	0.05	0.09	0.05	0.04	0.07	1.205
6510-10	9/25/01	0.06	0.06	0.11	0.04	0.04	0.07	1.8
6510-11	10/9/01	0.08	0.04	0.08	0.05	0.06	0.06	2.0
6510-12	10/22/01	0.04	0.05	0.04	0.05	0.06	0.05	1.96

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/8/2014	DIASORIN ANTI-Hbe TEST DATE: N/A	ORTHO VITROS ECi ANTI-Hbe TEST DATE: 8/8/2014	ABBOTT ANTI-HBs QUANT TEST DATE: N/A	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/8/2014	ORTHO VITROS ECi HBeAg TEST DATE: 8/8/2014	DIASORIN HBeAg TEST DATE: N/A
6510-01	3.76	2.5	0.00	0.2	1.4	2090	21.8
6510-02	3.26	2.7	0.00	0.3	0.9	2180	21.0
6510-03	2.51	1.7	0.09	0.3	1.4	23.5	6.8
6510-04	2.26	1.8	0.11	0.3	0.7	14.2	5.6
6510-05	2.28	1.6	0.12	0.15	0.9	14.3	17.2
6510-06	2.23	0.2	16.1	0.4	3.3	0.11	0.02
6510-07	1.57	0.1	15.0	0.84	8.9	0.11	0.08
6510-08	1.45	0.1	12.6	1.54	18.1	0.11	0.02
6510-09	1.36	0.2	6.54	3.4	QNS	0.10	0.16
6510-10	1.17	0.2	4.26	6.2	61.8	0.08	0.06
6510-11	1.15	0.3	5.12	5.7	111	0.10	0.05
6510-12	1.16	0.2	6.18	6.0	169	0.12	0.02

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6510 Rev03
Effective Date: 09/01/2021