

# ZeptoMetrix®

## HBV Longitudinal Panel Part Number: HBV6517

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

**\*Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/8/2014	TEST DATE: 8/8/2014	TEST DATE: N/A	TEST DATE: 8/8/2014	TEST DATE: N/A	TEST DATE: 8/8/2014	TEST DATE: N/A
6517-01	6/21/01	0.10	29.3	>40.8	3030	0.06	0.03	>6.4
6517-02	7/4/01	0.08	29.4	0.02	0.11	0.05	0.03	>6.4
6517-03	7/19/01	0.08	28.3	0.31	0.11	0.02	0.03	>6.0
6517-04	8/1/01	0.09	28.1	0.05	0.06	0.02	0.02	>6.0
6517-05	8/15/01	0.07	28.0	0.05	0.06	0.07	0.02	5.0
6517-06	8/30/01	0.08	26.2	0.04	0.06	0.05	0.01	3.2
6517-07	9/13/01	0.07	25.7	0.02	0.12	0.04	0.02	2.0
6517-08	9/27/01	0.07	23.9	0.02	0.10	0.06	0.02	1.6
6517-09	10/11/01	0.12	24.5	0.17	0.09	0.06	0.04	1.8
6517-10	10/25/01	0.10	33.7	0.04	0.06	0.06	0.03	1.4
6517-11	11/8/01	0.08	34.2	0.08	0.06	0.06	0.02	1.3
6517-12	11/21/0	0.07	34.6	0.08	0.06	0.05	0.03	1.3
6517-13	12/4/01	0.14	34.6	0.05	0.08	0.04	0.04	1.2

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-Hbe	ORTHO VITROS ECi ANTI-Hbe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/8/2014	TEST DATE: N/A	TEST DATE: 8/8/2014	TEST DATE: N/A	TEST DATE: 8/8/2014	TEST DATE: 8/8/2014	TEST DATE: N/A
6517-01	3.80	0.5	0.92	0.3	2.1	0.36	0.4
6517-02	3.77	0.2	2.5	0.8	3.5	0.18	0.1
6517-03	3.6	0.06	13.5	1.25	3.5	0.14	0.3
6517-04	2.69	0.05	23.1	1.77	9.8	0.13	0.3
6517-05	2.03	0.01	34.7	3.1	11.7	0.11	0.4
6517-06	1.59	0.02	22.5	5.0	33.6	0.13	0.2
6517-07	0.99	0.01	23.7	9.9	106	0.17	0.3
6517-08	0.83	0.02	25.5	7.5	121	0.17	0.2
6517-09	0.76	0.02	27.4	16.0	226	0.14	0.3
6517-10	0.60	0.02	27.9	15.7	274	0.14	0.2
6517-11	0.62	0.02	25.1	15.7	394	0.15	0.3
6517-12	0.61	0.03	4.34	20.9	416	0.16	0.3
6517-13	0.45	0.02	26.2	>21.0	581	0.15	0.3

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6517 Rev02  
Effective Date: 09/01/2021