

# ZeptoMetrix®

## HBV Longitudinal Panel Part Number: HBV6530

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS Eci ANTI-HIV ½	ORTHO VITROS Eci ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS Eci HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS Eci ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK
6530-01	10/18/01	0.07	0.07	>36.4	591	0.02	0.07	>4.7
6530-02	11/01/01	0.07	0.04	>36.4	3350	0.04	0.02	>4.7
6530-03	11/15/01	0.08	0.03	>36.4	6300	0.04	0.02	>4.7
6530-04	11/27/01	0.07	0.05	>36.4	9360	0.02	0.02	>4.7
6530-05	12/11/01	0.07	0.04	>36.4	7760	0.04	0.01	4.0
6530-06	12/25/01	0.08	0.03	>36.4	3410	0.04	0.01	3.0
6530-07	01/08/02	QNS	QNS	0.7	QNS	0.06	QNS	3.3
6530-08	01/22/02	0.09	0.03	0.2	0.07	0.04	0.01	1.9
6530-09	02/05/02	0.08	0.04	0.05	0.07	0.04	0.02	1.9
6530-10	02/19/02	0.08	0.03	0.1	0.06	0.05	0.01	1.8
6530-11	03/05/02	0.08	0.04	14.0	0.06	0.07	0.01	1.6
6530-12	03/18/02	0.06	0.02	0.2	0.09	0.07	0.01	1.4

PANEL NUMBER	ORTHO VITROS Eci ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS Eci ANTI-Hbe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS Eci ANTI-HBs QUANT	ORTHO VITROS Eci HBeAg	DIASORIN HBeAg
	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK
6530-01	3.79	0.06	2.17	0.3	2.1	0.27	0.10
6530-02	3.67	0.01	23.5	0.3	1.3	0.16	0.22
6530-03	3.36	0.01	26.5	0.4	2.3	0.15	0.09
6530-04	2.70	0.01	27.3	0.3	1.7	0.14	0.10
6530-05	1.89	0.01	31.6	0.3	2.5	0.14	0.05
6530-06	1.49	0.02	27.1	0.3	1.7	0.13	0.13
6530-07	QNS	0.05	QNS	0.4	QNS	QNS	0.09
6530-08	0.91	0.01	30.7	0.5	3.7	0.13	0.09
6530-09	0.87	0.01	31.9	0.5	3.6	0.12	0.05
6530-10	0.76	0.01	20.6	0.7	5.9	0.14	0.16
6530-11	0.82	0.01	27.0	0.6	13.5	0.14	0.08
6530-12	0.70	0.01	19.1	0.8	15.5	0.16	0.19

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6530 Rev03  
Effective Date: 09/07/2021