



**HBV Longitudinal Panel  
Part Number: HBV6534**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

**\*Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK
6534-01	11/12/01	0.12	36.6	3.7	9.58	0.02	0.03	>4.7
6534-02	11/26/01	0.31	36.9	8.2	0.12	0.02	0.06	>4.7
6534-03	12/11/01	0.26	37.2	0.2	0.09	0.04	0.03	>4.7
6534-04	12/25/01	0.12	36.3	0.2	0.09	0.02	0.03	>4.7
6534-05	01/08/02	0.07	36.0	0.2	0.13	0.03	0.03	>4.7
6534-06	01/22/02	0.08	36.2	0.2	0.09	0.04	0.02	>4.7
6534-07	02/05/02	0.09	35.8	0.2	0.08	0.03	0.02	>4.7
6534-08	02/19/02	0.10	35.6	0.2	0.07	0.05	0.02	>4.7
6534-09	03/05/02	0.12	35.9	0.2	0.07	0.03	0.02	>4.7
6534-10	03/19/02	0.08	35.7	0.2	0.08	0.03	0.02	>4.7
6534-11	04/02/02	0.07	34.9	0.2	0.07	0.04	0.02	4.5
6534-12	04/16/02	0.10	34.8	0.2	0.08	0.04	0.02	4.4

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK
6534-01	3.63	0.4	0.36	0.5	2.9	0.91	1.74
6534-02	3.60	0.3	1.97	0.5	1.1	0.14	0.48
6534-03	QNS	0.1	17.6	1.0	QNS	0.11	0.16
6534-04	3.12	0.06	18.5	2.1	19.8	0.13	0.05
6534-05	2.75	0.06	12.0	2.8	58.4	0.18	0.10
6534-06	2.59	0.06	12.4	5.5	116	0.15	0.04
6534-07	2.63	0.07	15.0	7.8	140	0.13	0.06
6534-08	2.50	0.06	12.6	6.1	143	0.14	0.09
6534-09	2.45	0.06	16.7	9.8	180	0.12	0.06
6534-10	2.37	0.05	15.7	8.9	148	0.16	0.03
6534-11	2.36	0.04	12.8	8.4	147	0.14	0.10
6534-12	2.26	0.05	12.5	9.4	146	0.14	0.03

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)