

ZeptoMetrix®

HBV Longitudinal Panel Part Number: HBV6535

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

***Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECI ANTI-HIV ½ TEST DATE: 8/21/2014	ORTHO VITROS ECI ANTI-HCV TEST DATE: 8/21/2014	ABBOTT HBsAg TEST DATE: UNK	ORTHO VITROS ECI HBsAg TEST DATE: 8/21/2014	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS ECI ANTI-HBc TOTAL TEST DATE: 8/21/2014	ABBOTT ANTI-HBc IgM TEST DATE: UNK
6535-01	11/13/01	0.09	32.8	>36.4	9260	0.03	0.09	>4.7
6535-02	11/26/01	0.10	33.4	>36.4	10200	0.04	0.14	>4.7
6535-03	12/11/01	0.09	32.7	>36.4	1720	0.05	0.18	>4.7
6535-04	12/25/01	0.11	32.6	1.8	1.47	0.04	0.13	>4.7
6535-05	01/08/02	0.07	32.5	0.2	0.07	0.06	0.08	4.6
6535-06	01/22/02	0.10	33.6	0.2	0.07	0.04	0.03	3.0
6535-07	02/05/02	0.10	32.8	0.1	0.06	0.06	0.02	2.0
6535-08	02/19/02	0.09	33.0	0.1	0.07	0.06	0.01	1.8
6535-09	03/05/02	0.07	32.3	0.2	0.06	0.06	0.01	2.1
6535-10	03/19/02	0.09	32.8	0.2	0.05	0.06	0.01	1.8
6535-11	04/02/02	0.09	33.6	0.2	0.07	0.05	0.02	1.4
6535-12	04/16/02	0.10	32.9	0.1	0.05	0.09	0.02	1.6

PANEL NUMBER	ORTHO VITROS ECI ANTI-HBc IgM TEST DATE: 8/21/2014	DIASORIN ANTI-HBe TEST DATE: UNK	ORTHO VITROS ECI ANTI-HBe TEST DATE: 8/21/2014	ABBOTT ANTI-HBs QUANT TEST DATE: UNK	ORTHO VITROS ECI ANTI-HBs QUANT TEST DATE: 8/21/2014	ORTHO VITROS ECI HBeAg TEST DATE: 8/21/2014	DIASORIN HBeAg TEST DATE: UNK
6535-01	3.62	0.09	5.68	0.3	3.3	0.15	0.13
6535-02	3.75	0.1	2.12	0.3	3.6	0.16	0.06
6535-03	3.39	0.03	13.4	0.3	2.8	0.14	0.09
6535-04	2.80	0.02	9.01	0.3	2.4	0.42	0.06
6535-05	2.02	0.02	28.7	0.3	3.4	0.15	0.08
6535-06	1.37	0.01	23.4	0.6	4.8	0.14	0.01
6535-07	1.10	0.01	32.6	0.7	7.2	0.13	0.05
6535-08	0.88	0.01	28.8	0.8	9.9	0.13	0.03
6535-09	0.80	0.01	33.5	1.0	13.8	0.12	0.08
6535-10	0.73	0.01	34.2	1.0	12.4	0.13	0.03
6535-11	0.76	0.03	25.7	1.3	12.7	0.16	0.14
6535-12	0.68	0.01	32.0	1.2	11.6	0.13	0.08

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6535 Rev03
Effective Date: 09/07/2021