

# ZeptoMetrix®

## HBV Longitudinal Panel Part Number: HBV6536

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

**\*Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK
6536-01	11/13/01	0.10	34.9	>36.4	8340	0.04	0.09	>4.7
6536-02	11/26/01	0.07	34.8	>36.4	2060	0.03	0.14	>4.7
6536-03	12/11/01	0.09	35.2	>36.4	44.9	0.07	0.11	>4.7
6536-04	12/25/01	0.08	34.7	5.6	3.57	0.05	0.04	>4.7
6536-05	01/08/02	0.11	36.6	0.4	0.13	0.11	0.04	3.9
6536-06	01/22/02	0.11	35.6	0.7	0.10	0.09	0.02	3.1
6536-07	02/05/02	0.10	34.8	0.2	0.10	0.06	0.03	2.5
6536-08	02/19/02	0.09	34.9	0.2	0.09	0.07	0.02	2.4
6536-09	03/05/02	0.10	35.0	0.2	0.09	0.07	0.03	2.1
6536-10	03/19/02	0.11	35.4	0.2	0.09	0.07	0.02	1.8
6536-11	04/02/02	0.09	35.2	0.1	0.09	0.07	0.02	2.0
6536-12	04/16/02	0.10	35.2	0.2	0.10	0.06	0.02	1.7

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK
6536-01	3.70	0.3	0.97	0.3	1.6	0.40	0.31
6536-02	3.66	0.4	1.45	0.3	1.7	0.14	0.31
6536-03	3.02	0.1	9.54	0.4	2.5	0.14	0.34
6536-04	2.37	0.07	26.8	0.3	2.7	0.12	0.26
6536-05	1.66	0.05	20.6	0.5	3.5	0.20	0.07
6536-06	1.43	0.05	26.7	0.7	5.2	0.21	0.16
6536-07	1.28	0.03	27.7	0.9	8.5	0.19	0.09
6536-08	1.21	0.04	8.28	1.3	12.1	0.19	0.23
6536-09	1.10	0.04	26.9	1.8	19.5	0.19	0.16
6536-10	1.01	0.05	25.6	2.2	20.3	0.17	0.26
6536-11	1.04	0.05	25.7	2.2	33.1	0.18	0.16
6536-12	0.99	0.05	19.2	2.1	26.6	0.19	0.24

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6536 Rev02  
Effective Date: 09/07/2021