

ZeptoMetrix®

HBV Longitudinal Panel Part Number: HBV6541

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS Eci ANTI-HIV ½	ORTHO VITROS Eci ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS Eci HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS Eci ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK
6541-01	12/03/01	0.06	0.05	>37	6270	0.1	0.04	>6.5
6541-02	12/17/01	0.09	0.04	>37	9590	0.1	0.04	>6.5
6541-03	12/31/01	0.09	0.03	>37	1990	0.1	0.03	>6.5
6541-04	01/14/02	0.09	0.04	1.1	0.85	0.1	0.04	6.0
6541-05	01/28/02	0.10	0.04	0.1	0.13	0.1	0.02	3.9
6541-06	02/11/02	0.09	0.04	0.1	0.06	0.1	0.02	3.3
6541-07	02/25/02	0.08	0.05	0.2	0.24	0.3	0.01	3.1
6541-08	03/11/02	0.10	0.04	0.1	0.06	0.1	0.01	2.6
6541-09	03/25/02	0.09	0.03	0.1	0.05	0.1	0.01	1.9
6541-10	04/08/02	0.14	0.03	0.1	0.06	0.1	0.02	1.8
6541-11	04/22/02	0.12	0.03	0.1	0.06	0.1	0.02	1.4
6541-12	05/06/02	0.11	0.04	0.1	0.06	0.1	0.02	1.6

PANEL NUMBER	ORTHO VITROS Eci ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS Eci ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS Eci ANTI-HBs QUANT	ORTHO VITROS Eci HBeAg	DIASORIN HBeAg
	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK
6541-01	3.78	0.5	0.25	0.3	1.6	2.54	1.8
6541-02	3.68	0.3	0.48	0.5	2.0	0.32	0.39
6541-03	3.38	0.3	1.21	0.5	1.4	0.14	0.12
6541-04	2.79	0.1	4.03	0.5	2.5	0.15	0.12
6541-05	2.13	0.1	5.64	0.5	2.6	0.27	0.16
6541-06	1.76	0.09	12.5	0.8	4.6	0.15	0.11
6541-07	1.73	0.09	15.6	1.08	12.8	0.16	0.34
6541-08	1.29	0.08	28.4	2.5	21.5	0.14	0.18
6541-09	1.09	0.06	14.6	2.3	32.8	0.13	0.04
6541-10	1.00	0.07	16.9	2.5	54.6	0.15	0.16
6541-11	0.94	0.05	15.5	3.3	57.0	0.15	0.12
6541-12	0.91	0.05	18.8	2.2	75.0	0.14	0.09

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6541 Rev03
Effective Date: 06/14/2021