

# ZeptoMetrix<sup>®</sup>

## HBV Longitudinal Panel Part Number: HBV6545

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

**\*Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: N/A	TEST DATE: 8/21/2014	TEST DATE: N/A	TEST DATE: 8/21/2014	TEST DATE: N/A
6545-01	1/23/02	0.11	19.9	>36.4	3630	0.04	0.07	>5.5
6545-02	2/6/02	0.10	19.2	>36.4	7350	0.03	0.08	>5.5
6545-03	2/19/02	0.11	18.9	>36.4	9550	0.1	0.05	>5.5
6545-04	3/6/02	0.12	19.1	>36.4	8510	0.5	0.03	>5.5
6545-05	3/19/02	0.11	20.4	>36.4	3420	0.1	0.02	4.7
6545-06	4/2/02	0.11	22.1	>36.4	1170	0.04	0.02	4.1
6545-07	4/16/02	0.10	22.4	>36.4	300	0.1	0.02	4.0
6545-08	4/30/02	0.11	22.1	>36.4	68.9	0.1	0.02	3.7
6545-09	5/14/02	0.13	21.4	24.2	12.9	0.1	0.02	3.4
6545-10	5/28/02	0.11	21.8	5.6	1.57	0.1	0.02	3.1
6545-11	6/11/02	0.10	22.1	1.7	0.64	0.1	0.02	3.2
6545-12	6/25/02	0.10	21.5	0.5	0.09	0.1	0.03	2.9

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-Hbe	ORTHO VITROS ECi ANTI-Hbe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/21/2014	TEST DATE: N/A	TEST DATE: 8/21/2014	TEST DATE: N/A	TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: N/A
6545-01	3.70	0.2	2.60	0.3	2.6	0.23	0.11
6545-02	3.40	0.2	1.52	0.3	1.4	0.16	0.16
6545-03	3.06	0.2	3.20	0.3	1.9	0.15	0.05
6545-04	2.60	0.07	6.13	0.5	1.4	0.21	0.18
6545-05	2.21	0.04	9.43	0.4	1.2	0.15	0.04
6545-06	1.91	0.04	12.6	0.4	1.6	0.16	0.12
6545-07	1.88	0.04	15.6	0.4	3.1	0.14	0.04
6545-08	1.66	0.05	9.93	0.3	1.2	0.16	0.14
6545-09	1.47	0.04	19.0	0.5	1.0	0.13	0.07
6545-10	1.43	0.09	15.3	0.4	1.5	0.15	0.16
6545-11	1.42	0.04	15.9	0.3	1.8	0.16	0.08
6545-12	1.34	0.05	16.1	0.3	1.6	0.15	0.11

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6545 Rev02  
Effective Date: 09/22/2021