



**HBV Longitudinal Panel
Part Number: HBV6548**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures .*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ABBOTT ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM	DIASORIN ANTI-HBe	ABBOTT ANTI-HBs QUANT	DIASORIN HBeAg
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6548-01	02/08/02	0.06	0.04	>36.4	0.04	3.6	0.2	0.3	0.18
6548-02	02/22/02	0.06	0.02	1.0	0.05	5.1	0.3	0.3	0.16
6548-03	03/09/02	0.08	0.02	0.05	0.05	>5.5	0.2	0.4	0.10
6548-04	03/23/02	0.07	0.02	0.07	0.06	>5.5	0.1	0.5	0.12
6548-05	04/06/02	0.08	0.03	0.8	0.06	>5.5	0.08	1.04	0.04
6548-06	04/20/02	0.06	0.03	0.04	0.05	>5.5	0.08	1.5	0.12
6548-07	05/04/02	0.08	0.02	0.04	0.07	>5.5	0.07	1.4	0.04
6548-08	05/18/02	0.07	0.03	0.04	0.06	>5.5	0.07	1.9	0.08
6548-09	06/01/02	0.08	0.02	0.04	0.06	>5.5	0.07	2.5	0.04
6548-10	06/15/02	0.07	0.02	0.02	0.06	>5.5	0.05	1.4	0.10
6548-11	06/29/02	0.06	0.03	0.7	0.05	5.3	0.07	1.3	0.05
6548-12	07/13/02	0.08	0.02	0.05	0.07	>5.5	0.08	2.3	0.10

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6548 Rev02
Effective Date: 09/22/2021