

# ZeptoMetrix®

## EBV Seroconversion Panel Donor No.283843421 Part Number: EBV9097

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BLEED NUMBER	DATE OF DRAW	EA D IgM	EA D IgG	EBNA-1 IgM	EBNA-1 IgG	VCA IgM	VCA IgG
			TEST DATE: 08/19/2009	TEST DATE: 08/19/2009	TEST DATE: 08/19/2009	TEST DATE: 08/19/2009	TEST DATE: 08/19/2009	TEST DATE: 08/19/2009
9097-01	8520017029	12/18/2008	0.464	0.415	0.385	0.395	0.175	0.207
9097-02	8520017216	12/20/2008	0.411	0.374	0.249	0.331	0.117	0.172
9097-03	8520018960	01/15/2009	0.391	0.378	0.223	0.374	0.137	0.257
9097-04	8520019100	01/17/2009	0.299	0.429	0.219	0.387	0.116	0.214
9097-05	8520019667	01/24/2009	0.267	0.354	0.190	0.285	0.106	0.220
9097-06	8520020873	02/12/2009	2.736	1.956	0.524	0.394	2.243	1.192
9097-07	8520021550	02/21/2009	2.235	1.752	0.909	0.483	2.036	1.993
9097-08	8520021928	02/26/2009	1.717	1.715	0.776	0.510	1.598	2.092
9097-09	8520022083	02/28/2009	1.712	1.692	0.709	0.468	1.508	2.174
9097-10	8520030035	06/04/2009	0.532	0.648	0.428	3.694	0.373	2.806
9097-11	8500030257	06/06/2009	0.557	0.616	0.422	3.532	0.352	2.594
9097-12	8520030631	06/11/2009	0.519	0.615	0.402	3.643	0.342	2.679
9097-13	8500030853	06/13/2009	0.489	0.587	0.390	3.645	0.324	2.720
9097-14	8520031330	06/18/2009	0.557	0.657	0.453	3.683	0.363	2.759

PANEL MEMBER	BLEED NUMBER	DATE OF DRAW	MONO SCREEN	ASI TITER	ASI MONO
			TEST DATE: 08/19/2009	TEST DATE: 08/19/2009	TEST DATE: 08/19/2009
9097-01	8520017029	12/18/2008	NEGATIVE	0.500	NEGATIVE
9097-02	8520017216	12/20/2008	NEGATIVE	0.500	NEGATIVE
9097-03	8520018960	01/15/2009	NEGATIVE	0.500	NEGATIVE
9097-04	8520019100	01/17/2009	NEGATIVE	0.500	NEGATIVE
9097-05	8520019667	01/24/2009	NEGATIVE	0.500	NEGATIVE
9097-06	8520020873	02/12/2009	POSITIVE	4.000	POSITIVE
9097-07	8520021550	02/21/2009	POSITIVE	4.000	POSITIVE
9097-08	8520021928	02/26/2009	POSITIVE	4.000	POSITIVE
9097-09	8520022083	02/28/2009	POSITIVE	5.000	POSITIVE
9097-10	8520030035	06/04/2009	POSITIVE	2.000	POSITIVE
9097-11	8500030257	06/06/2009	POSITIVE	2.000	POSITIVE
9097-12	8520030631	06/11/2009	POSITIVE	2.000	POSITIVE
9097-13	8500030853	06/13/2009	POSITIVE	2.000	POSITIVE
9097-14	8520031330	06/18/2009	POSITIVE	2.000	POSITIVE

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIEBV9097 Rev02  
Effective Date: 06/21/2021