

ZeptoMetrix®

HBV Seroconversion Panel Donor No.64090 Part Number: HBV10231

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	VENDOR ALT	ZEPTOMETRIX HBV RT-PCR Copies/mL	VENDOR HBsAg	VENDOR HIV p24Ag	GENETICS SYSTEMS HBsAg	ABBOTT Auszyme Monoclonal HBsAg
			TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
10231-01	04/26/97	7	Not Detected	Negative	Negative	Negative	0.100
10231-02	05/13/97	15	102	Negative	Negative	Negative	0.117
10231-03	06/02/97	11	905	Negative	Negative	Negative	0.267
10231-04	06/08/97	10	33.3	Positive	Negative	Positive	1.512
10231-05	06/17/97	10	2,960	Positive	Negative	Positive	4.667

PANEL MEMBER	ORTHO VITROS ECI HBsAg	ORTHO VITROS ECI Anti-HBc Total	ORTHO VITROS ECI HBeAg	ORTHO VITROS ECI Anti-HBe	ORTHO VITROS ECI Anti-HBc IgM	DIASORIN Anti-HBs	DIASORIN HBc IgM	ABBOTT Anti-HBc Total
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
10231-01	1.41	3.49	0.18	0.16	0.00	0.03	0.17	0.495
10231-02	0.15	3.73	0.16	0.18	0.00	0.02	0.17	0.521
10231-03	0.42	3.74	0.16	0.20	0.01	0.05	0.17	0.485
10231-04	3.02	3.69	0.18	0.18	0.01	0.01	0.17	0.496
10231-05	18.8	3.67	0.22	0.19	0.01	0.04	0.17	0.581

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV10231 Rev02
Effective Date: 08/11/2021