

ZeptoMetrix®

HBV Seroconversion Panel Donor No. 65449 Part Number: HBV11000

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL NUMBER | DATE OF DRAW | ALT Normal Range (10-45u/L) | ABBOTT HBV REAL TIME CP/ML | GENETIC SYSTEMS HBsAg (2.0) | ABBOTT Auszyme MC Dynamic HBsAg | ABBOTT Auszyme Monoclonal HBsAg | Abbott MUREX HBsAg Version 3 | ORTHO HBsAg ELISA Testsystem 3 | ROCHE ELECSYS HBsAg |
|--------------|--------------|-----------------------------------|-------------------------------------|--------------------------------------|--|--|---------------------------------------|---|---------------------------|
| | | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK |
| 11000-01 | 10/30/97 | 3 | 1210 | 0.449 | 0.200 | 0.094 | 0.68 | 0.414 | 0.450 |
| 11000-02 | 11/02/97 | 3 | 650 | 0.449 | 0.200 | 0.075 | 0.67 | 0.173 | 0.510 |
| 11000-03 | 11/11/97 | 5 | 2772 | 0.449 | 0.260 | 0.189 | 0.83 | 0.147 | 0.530 |
| 11000-04 | 11/13/97 | 4 | 3739 | 0.577 | 0.470 | 0.472 | 1.02 | 0.388 | 0.700 |
| 11000-05 | 11/18/97 | 4 | 4817 | 0.513 | 0.520 | 0.887 | 1.24 | 0.441 | 0.870 |
| 11000-06 | 11/20/97 | 5 | 6064 | 0.285 | 0.800 | 0.981 | 1.55 | 0.655 | 1.200 |
| 11000-07 | 11/25/97 | 3 | 13892 | 0.256 | 1.390 | 2.547 | 2.70 | 1.191 | 1.930 |
| 11000-08 | 11/28/97 | 5 | 12964 | 0.641 | 1.940 | 2.849 | 3.23 | 1.750 | 2.850 |
| 11000-09 | 12/02/97 | 7 | 36539 | 3.462 | 9.760 | 7.189 | 10.51 | 7.799 | 9.530 |

| PANEL NUMBER | ORTHO VITROS ECi HBsAg | ABBOTT PRISM HBsAg | DIASORIN HBsAg | DADE BEHRING Enzygnost HBsAg 5.0 | ORTHO VITROS HBsAg Reagent Pack | ABBOTT AxSYM HBsAg (V2) Reagent Pack | SORIN ETI-MAK 4 | ORTHO VITROS ECi ANTI-HBs QUANT | DIASORIN ANTI-HBs QUANT |
|--------------|---------------------------------|--------------------------|-------------------|---|--|---|-----------------------|--|-------------------------------|
| | TEST DATE: 8/14/2015 | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 8/14/2015 | TEST DATE: UNK |
| 11000-01 | 0.17 | 0.49 | 0.195 | 0.280 | 0.140 | 0.550 | 0.330 | 33.3 | 4.11 |
| 11000-02 | 0.16 | 0.52 | 0.176 | 0.280 | 0.130 | 0.610 | 0.360 | 26.5 | 3.16 |
| 11000-03 | 0.14 | 1.11 | 0.155 | 0.280 | 0.120 | 0.640 | 0.430 | 16.4 | 2.34 |
| 11000-04 | 0.15 | 1.23 | 0.184 | 0.320 | 0.180 | 0.780 | 0.500 | 12.8 | 1.52 |
| 11000-05 | 0.17 | 1.51 | 0.168 | 0.370 | 0.240 | 0.900 | 0.620 | 10.1 | 1.12 |
| 11000-06 | 0.17 | 2.05 | 0.187 | 0.480 | 0.390 | 1.030 | 0.690 | 8.4 | 0.93 |
| 11000-07 | 0.33 | 4.36 | 0.201 | 1.530 | 0.950 | 1.820 | 1.520 | 3.9 | 0.39 |
| 11000-08 | 0.77 | 5.24 | 0.254 | 1.660 | 1.600 | 2.390 | 2.110 | 2.0 | 0.21 |
| 11000-09 | 10.1 | 14.37 | 0.885 | 9.700 | 11.080 | 5.570 | 11.010 | 0.5 | 0.05 |

| PANEL NUMBER | ORTHO VITROS ECi HBeAg | ABBOTT HBeAg (Co/S) | ORTHO VITROS ECi ANTI-HBe | ORTHO VITROS ECi ANTI-HBc IgM | DIASORIN HBc IgM | ABBOTT ANTI-HBc TOTAL | ORTHO VITROS ECi ANTI-HBc TOTAL | DIASORIN ANTI-HBc TOTAL | ORTHO VITROS ECi ANTI-HAV TOTAL |
|--------------|---------------------------------|---------------------------|------------------------------------|-------------------------------------|---------------------|-----------------------------|--|-------------------------------|--|
| | TEST DATE: 8/14/2015 | TEST DATE: UNK | TEST DATE: 8/14/2015 | TEST DATE: 8/14/2015 | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 8/14/2015 | TEST DATE: UNK | TEST DATE: 8/14/2015 |
| 11000-01 | 0.23 | Not Tested | 0.15 | 0.02 | 0.2 | Not Tested | 2.87 | 4.33 | 3.02 |
| 11000-02 | 0.24 | Not Tested | 0.16 | 0.01 | 0.2 | Not Tested | 3.00 | 4.27 | 3.17 |
| 11000-03 | 0.25 | Not Tested | 0.18 | 0.02 | 0.2 | Not Tested | 2.77 | 4.93 | 2.93 |
| 11000-04 | 0.23 | Not Tested | 0.15 | 0.01 | 0.2 | Not Tested | 2.75 | 3.33 | 2.91 |
| 11000-05 | 0.25 | Not Tested | 0.15 | 0.03 | 0.2 | Not Tested | 2.32 | 4.67 | 2.24 |
| 11000-06 | 0.24 | Not Tested | 0.15 | 0.02 | 0.2 | Not Tested | 2.47 | 4.53 | 2.43 |
| 11000-07 | 0.26 | Not Tested | 0.15 | 0.01 | 0.2 | Not Tested | 2.92 | 4.47 | 2.78 |
| 11000-08 | 0.28 | Not Tested | 0.16 | 0.01 | 0.2 | Not Tested | 3.04 | 4.33 | 3.00 |
| 11000-09 | 0.35 | 0.338 | 0.18 | 0.01 | 0.2 | 0.395 | 3.05 | 4.07 | 3.16 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11000 Rev03
Effective Date: 08/11/2021