

ZeptoMetrix®

HBV Seroconversion Panel Donor No. 65524 Part Number: HBV11001

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 3/15/1999	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: UNK	ROCHE Cobas Core HBsAg II EIA TEST DATE: UNK	DIASORIN HBsAg TEST DATE: UNK	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: UNK
11001-01	10/12/97	2	NOT DETECTED	Not Tested	0.300	0.200	0.206	0.513
11001-02	10/18/97	4	NOT DETECTED	Not Tested	0.190	0.180	0.209	0.577
11001-03	11/14/97	7	73	NEG	0.190	0.190	0.163	0.449
11001-04	11/16/97	4	156	3280	0.080	0.230	0.246	0.577
11001-05	11/25/97	2	1358	5330	2.780	1.490	0.270	0.577
11001-06	11/28/97	3	2709	11700	2.320	2.510	0.401	1.667
11001-07	12/10/97	1	107834	Not Tested	59.140	69.600	4.888	9.167
11001-08	12/14/97	4	303917	832000	OVER	OVER	5.168	10.449

PANEL MEMBER	DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 1/24/2000	ORTHO VITROS HBsAg Reagent Pack TEST DATE: 2/24/2000	ROCHE Elecys HBsAg TEST DATE: 2/9/2000	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 3/22/2000	SORIN ETI-MAK 4 TEST DATE: UNK	ABBOTT PRISM HBsAg TEST DATE: 8/27/2009	ABBOTT MUREX HBsAg Version 3 TEST DATE: 8/27/2009	ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 2/28/2000	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK
11001-01	0.340	0.090	0.520	0.190	0.150	0.77	0.51	0.370	0.075
11001-02	0.290	0.090	0.540	0.190	0.150	0.52	0.50	0.340	0.057
11001-03	0.260	0.090	0.580	0.190	0.250	0.74	0.61	0.390	0.113
11001-04	0.350	0.090	0.610	0.320	0.280	0.63	0.64	0.420	0.113
11001-05	1.600	1.840	1.800	2.080	1.830	4.50	2.53	0.920	1.642
11001-06	2.810	3.340	2.950	2.750	3.330	8.36	4.37	1.540	3.226
11001-07	35.030	144.000	72.490	OVER	70.150	141.70	31.27	33.340	>37.736
11001-08	OVER	521.000	258.500	OVER	87.250	44.92	32.01	60.870	>37.736

PANEL MEMBER	ORTHO VITROS ECi HBsAg TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL (OD) TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS ECi HBeAg TEST DATE: UNK	ABBOTT HBeAg (OD) TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBe TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: UNK
11001-01	0.14	Not Tested	0.01	3.06	0.22	Not Tested	0.17	1.4
11001-02	0.13	Not Tested	0.01	3.07	0.23	Not Tested	0.15	0.5
11001-03	0.14	Not Tested	0.01	3.21	0.22	Not Tested	0.14	1.7
11001-04	0.17	Not Tested	0.01	3.03	0.21	Not Tested	0.17	0.8
11001-05	1.66	Not Tested	0.01	3.01	0.23	Not Tested	0.15	1.8
11001-06	3.45	Not Tested	0.01	3.15	0.23	Not Tested	0.15	0.5
11001-07	118	Not Tested	0.02	3.08	0.55	Not Tested	0.15	1.4
11007-08	448	0.88	0.01	3.19	2.26	0.608	0.14	2.3

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11001 Rev03
Effective Date: 08/11/2021