



**HBV Seroconversion Panel Donor No. 67449  
Part Number: HBV11009**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 6/4/1999	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ABBOTT AxSYM HBsAg (V2) Reagent Pack  TEST DATE: UNK	ORTHO VITROS HBsAg Reagent Pack  TEST DATE: UNK	DADE BEHRING Enzygnost HBsAg 5.0  TEST DATE: 3/16/2000	GENETIC SYSTEMS HBsAg (2.0)  TEST DATE: 7/14/2000	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: 1/4/2000
11009-01	03/12/98	11	NOT DETECTED	0.350	0.300	0.190	0.438	0.000
11009-02	03/14/98	13	NOT DETECTED	0.490	0.300	0.220	0.563	0.040
11009-03	03/19/98	17	NOT DETECTED	0.500	0.320	0.220	0.500	0.067
11009-04	03/21/98	14	NOT DETECTED	0.480	0.330	0.090	0.438	0.040
11009-05	03/26/98	14	NOT DETECTED	0.420	0.330	0.120	0.438	0.040
11009-06	03/28/98	9	NOT DETECTED	0.470	0.290	0.120	0.313	0.000
11009-07	04/05/98	15	NOT DETECTED	0.430	0.310	0.140	0.313	0.000
11009-08	04/16/98	19	NOT DETECTED	0.420	0.310	0.230	0.750	0.000
11009-09	04/19/98	14	NOT DETECTED	0.430	0.320	0.230	0.688	0.000
11009-10	04/30/98	8	NOT DETECTED	0.440	0.310	0.260	0.500	0.000
11009-11	05/03/98	11	NOT DETECTED	0.510	0.290	0.190	0.500	0.000
11009-12	05/07/98	14	NOT DETECTED	0.540	0.310	0.140	0.438	0.000
11009-13	05/10/98	0	NOT DETECTED	0.440	0.330	0.150	0.563	0.000
11009-14	05/14/98	11	NOT DETECTED	0.470	0.310	0.200	0.500	0.000
11009-15	05/17/98	10	NOT DETECTED	0.520	0.330	0.170	0.438	0.520
11009-16	05/23/98	18	<52	0.470	0.350	0.220	0.438	0.600
11009-17	05/25/98	12	124	0.510	0.460	0.280	0.750	0.550
11009-18	05/30/98	26	1030	0.720	1.070	0.740	0.563	1.080
11009-19	06/01/98	21	2470	1.050	2.060	1.550	0.875	2.287
11009-20	06/06/98	15	20547	4.570	17.800	11.170	2.813	11.710
11009-21	06/08/98	18	59259	11.030	39.130	22.070	5.938	26.289
11009-22	06/13/98	32	649762	64.390	322.310	OVER	13.750	OVER
11009-23	06/15/98	23	1873934	103.550	789.670	OVER	15.000	OVER

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11009 Rev04  
Effective Date: 08/12/2021

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No. 67449 Part Number: HBV11009

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ROCHE ELECSYS HBsAg TEST DATE: UNK	ABBOTT PRISM HBsAg TEST DATE: UNK	DIASORIN HBsAg TEST DATE: 10/24/1999	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 12/20/1999	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK	SORIN ETI-MAK 4 TEST DATE: UNK	ORTHO VITROS Eci HBsAg TEST DATE: 8/30/2018
11009-01	0.48	0.92	0.152	0.57	0.090	0.077	0.26	0.16
11009-02	0.52	0.63	0.107	0.57	0.120	0.096	0.19	0.09
11009-03	0.46	0.96	0.099	0.64	0.090	0.019	0.24	0.08
11009-04	0.47	0.86	0.078	0.65	0.090	0.038	0.26	0.08
11009-05	0.44	0.49	0.075	0.63	0.090	0.038	0.19	0.09
11009-06	0.44	0.90	0.147	0.57	0.120	0.038	0.26	0.18
11009-07	0.51	0.35	0.123	0.65	0.120	NEGATIVE	0.21	0.09
11009-08	0.45	0.43	0.147	0.56	0.090	0.038	0.29	0.08
11009-09	0.48	0.55	0.115	0.59	0.200	0.038	0.26	0.08
11009-10	0.47	0.60	0.104	0.57	0.170	0.058	0.24	0.09
11009-11	0.49	0.70	0.128	0.65	0.120	0.077	0.21	0.08
11009-12	0.49	0.30	0.091	0.67	0.170	0.038	0.26	0.08
11009-13	0.48	0.26	0.115	0.63	0.120	0.019	0.21	0.09
11009-14	0.49	0.50	0.201	0.64	0.120	0.038	0.55	0.09
11009-15	0.44	0.80	0.152	0.58	0.120	NEGATIVE	0.21	0.09
11009-16	0.51	0.48	0.147	0.62	0.170	0.058	0.29	0.11
11009-17	0.49	1.14	0.160	0.64	0.260	0.038	0.33	0.14
11009-18	0.70	1.77	0.214	1.04	0.550	0.385	0.81	0.54
11009-19	1.14	3.33	0.206	1.87	1.250	0.769	1.55	1.52
11009-20	6.34	23.42	0.642	11.07	9.350	7.692	10.19	15.5
11009-21	14.09	49.04	1.302	21.57	23.590	15.962	20.87	33.8
11009-22	114.3	221.12	6.136	30.03	OVER	38.462	74.98	275
11009-23	290.5	329.09	4.578	29.21	OVER	38.462	82.98	641

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11009 Rev04  
Effective Date: 08/12/2021



**HBV Seroconversion Panel Donor No. 67449  
Part Number: HBV11009**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT HBeAg	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBs
	TEST DATE: 8/30/2018	TEST DATE: 10/25/1999	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11009-01	3.15	0.288	0.15	0.18	0.02	0.5
11009-02	3.17	0.288	0.14	0.17	0.02	0.6
11009-03	3.15	0.288	0.14	0.17	0.02	1.2
11009-04	3.20	0.288	0.14	0.17	0.02	0.0
11009-05	3.08	0.288	0.14	0.16	0.02	0.9
11009-06	3.01	0.192	0.13	0.17	0.02	0.3
11009-07	3.13	0.288	0.14	0.16	0.02	0.5
11009-08	3.27	0.288	0.13	0.19	0.02	0.4
11009-09	3.43	0.288	0.14	NOT TESTED	0.02	0.3
11009-10	3.42	0.288	0.14	0.17	0.01	0.6
11009-11	3.44	0.288	0.14	0.17	0.01	0.0
11009-12	3.47	0.288	0.13	0.17	0.01	0.3
11009-13	3.28	0.288	0.13	0.18	0.01	0.3
11009-14	3.16	0.481	0.15	0.17	0.02	0.7
11009-15	3.11	0.288	0.12	0.18	0.03	0.4
11009-16	3.19	0.288	0.13	0.18	0.02	1.2
11009-17	3.03	0.288	0.17	0.17	0.02	0.5
11009-18	3.01	0.288	0.18	0.15	0.02	0.0
11009-19	3.11	0.288	0.14	0.14	0.02	0.5
11009-20	3.14	2.692	0.18	0.15	0.02	0.1
11009-21	3.21	0.481	0.32	0.18	0.01	0.0
11009-22	3.24	1.250	5.08	0.15	0.01	0.0
11009-23	3.18	2.212	18.0	0.11	0.01	0.0

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)