



**HBV Seroconversion Panel Donor No. 68105**  
**Part Number: HBV11015**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT	ROCHE	GENETIC	ORTHO	ABBOTT	ABBOTT
		Normal Range (10-45u/L)	HBV REAL TIME CP/ML	CAP-G/CTM HBV (cp/mL)	SYSTEMS HBsAg (2.0)	HBsAg ELISA Testsystem 3	MUREX HBsAg Version 3	Auszyme MC Dynamic HBsAg
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11015-01	06/15/98	24	NOT DETECTED	NEGATIVE	0.571	0.000	0.55	0.060
11015-02	06/17/98	14	NOT DETECTED	NEGATIVE	0.500	0.000	0.69	0.200
11015-03	06/23/98	14	NOT DETECTED	NEGATIVE	0.714	0.000	0.54	0.520
11015-04	06/26/98	9	NOT DETECTED	NEGATIVE	0.714	0.000	0.55	0.200
11015-05	06/30/98	7	NOT DETECTED	NEGATIVE	0.643	0.000	0.53	0.520
11015-06	07/03/98	11	NOT DETECTED	NEGATIVE	0.500	0.000	0.52	0.630
11015-07	07/11/98	10	NOT DETECTED	NEGATIVE	0.500	0.000	0.54	0.370
11015-08	07/13/98	10	NOT DETECTED	NEGATIVE	0.571	0.000	0.51	0.550
11015-09	07/19/98	12	<52	NEGATIVE	0.500	0.020	0.53	0.200
11015-10	07/21/98	8	<52	NEGATIVE	0.714	0.000	0.63	1.030
11015-11	07/26/98	2	54	NEGATIVE	0.571	0.020	0.63	0.430
11015-12	07/28/98	NOT TESTED	152	NEGATIVE	0.571	0.310	0.73	1.060
11015-13	08/24/98	9	634972	1510000	20.000	OVER	28.61	OVER
11015-14	08/26/98	8	1238096	NOT TESTED	21.429	OVER	28.72	OVER

PANEL MEMBER	ABBOTT	ROCHE	BIORAD	DADE	BIOTEST	DIASORIN	ABBOTT	ABBOTT	ABBOTT
	Auszyme Monoclonal HBsAg	Cobas Core HBsAg II EIA	MONALISA HBsAg PLUS	BEHRING Enzygnost HBsAg 5.0	ELISA HBsAg	HBsAg	PRISM HBsAg	AxSYM HBsAg (V2) Reagent Pack	HBeAg
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11015-01	0.091	0.140	0.290	0.410	0.180	0.112	0.71	0.510	0.385
11015-02	0.091	0.140	0.240	0.280	0.180	0.128	0.34	0.530	0.385
11015-03	0.036	0.140	0.150	0.260	0.100	0.134	0.48	0.820	0.288
11015-04	0.073	0.140	0.220	0.250	0.140	0.139	0.73	0.450	0.288
11015-05	0.073	0.140	0.460	0.230	0.120	0.136	0.71	0.400	0.385
11015-06	0.055	0.120	0.310	0.260	0.100	0.393	0.54	0.440	0.288
11015-07	0.073	0.120	0.350	0.260	0.120	0.249	0.76	0.490	0.385
11015-08	0.055	0.140	0.510	0.300	0.180	0.123	0.60	0.520	0.288
11015-09	0.073	0.160	0.500	0.280	0.180	0.144	0.34	0.510	0.288
11015-10	0.073	0.160	0.370	0.260	0.200	0.152	0.41	0.700	0.288
11015-11	0.236	0.200	0.380	0.330	0.140	0.179	0.79	0.530	0.288
11015-12	0.109	0.240	0.430	0.400	0.120	0.142	1.62	0.560	0.288
11015-13	OVER	70.470	58.480	OVER	56.120	4.570	353.39	101.590	2.404
11015-14	OVER	70.470	59.420	OVER	56.390	4.307	440.31	116.410	3.462

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11015 Rev03  
 Effective Date: 08/12/2021



**HBV Seroconversion Panel Donor No. 68105  
Part Number: HBV11015**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SORIN ETI-MAK 4	SORIN ETI-MAK 3	ABBOTT ANTI-HBc TOTAL (Co/S)	ORTHO VITROS ECi HBsAg	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBs QUANT
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11015-01	0.180	0.060	NOT TESTED	0.06	3.27	0.14	0.19	0.01	0.0
11015-02	0.200	0.020	NOT TESTED	0.05	3.08	0.14	0.18	0.01	0.2
11015-03	0.200	0.040	NOT TESTED	0.08	3.23	0.14	0.19	0.01	0.0
11015-04	0.230	0.040	NOT TESTED	0.05	3.09	0.13	0.17	0.01	0.0
11015-05	0.200	0.000	NOT TESTED	0.17	3.32	0.14	0.16	0.01	0.0
11015-06	0.200	0.040	NOT TESTED	0.05	3.05	0.14	0.18	0.01	0.0
11015-07	0.180	0.060	NOT TESTED	0.06	3.10	0.13	0.19	0.02	0.0
11015-08	0.200	0.020	NOT TESTED	0.05	3.50	0.14	0.17	0.02	0.0
11015-09	0.230	0.110	NOT TESTED	0.15	3.26	0.13	0.21	0.01	0.0
11015-10	0.230	0.070	NOT TESTED	0.05	3.16	0.13	0.18	0.01	0.0
11015-11	0.250	0.110	NOT TESTED	0.06	3.05	0.13	0.18	0.01	0.3
11015-12	0.330	0.110	NOT TESTED	0.06	3.20	0.12	0.18	0.01	0.0
11015-13	84.180	OVER	NOT TESTED	587	3.18	10.2	0.13	0.01	0.0
11015-14	84.530	OVER	0.603	1070	3.21	21.1	0.10	0.01	0.0

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)