

ZeptoMetrix®

HBV Seroconversion Panel Donor No. 68739 Part Number: HBV11017

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ALT Normal Range (10-45u/L) | ABBOTT HBV REAL TIME CP/ML | ROCHE CAP-G/CTM HBV (cp/mL) | GENETIC SYSTEMS HBsAg (2.0) | DADE BEHRING Enzygnost HBsAg 5.0 | BIORAD MONALISA HBsAg PLUS | ROCHE Cobas Core HBsAg II EIA |
|--------------|--------------|-----------------------------------|-------------------------------------|--------------------------------------|--------------------------------------|---|-------------------------------------|--|
| | | TEST DATE: 10/27/1998 | TEST DATE: UNK | TEST DATE: 6/2/2005 | TEST DATE: UNK | TEST DATE: 12/15/1999 | TEST DATE: UNK | TEST DATE: 4/18/2000 |
| 11017-01 | 08/26/98 | 7 | NOT DETECTED | NEGATIVE | 0.643 | 0.770 | 0.430 | 0.180 |
| 11017-02 | 08/31/98 | 6 | NOT DETECTED | NEGATIVE | 0.571 | 0.210 | 0.380 | 0.160 |
| 11017-03 | 09/02/98 | 9 | NOT DETECTED | NEGATIVE | 0.643 | 0.320 | 0.500 | 0.160 |
| 11017-04 | 09/11/98 | 8 | NOT DETECTED | NEGATIVE | 0.714 | 0.190 | 0.570 | 0.180 |
| 11017-05 | 09/18/98 | 6 | NOT DETECTED | NEGATIVE | 0.500 | 0.290 | 0.330 | 0.180 |
| 11017-06 | 09/20/98 | 4 | <52 | NEGATIVE | 0.500 | 0.330 | 0.430 | 0.180 |
| 11017-07 | 09/27/98 | 8 | 66 | NEGATIVE | 0.571 | 0.300 | 0.280 | 0.220 |
| 11017-08 | 09/29/98 | 6 | 96 | 224 | 0.714 | 0.390 | 0.380 | 0.220 |
| 11017-09 | 10/05/98 | 13 | 318 | 619 | 0.643 | 0.610 | 0.600 | 0.480 |
| 11017-10 | 10/07/98 | 8 | 799 | 2230 | 0.643 | 0.870 | 0.820 | 0.660 |
| 11017-11 | 10/12/98 | 9 | 4100 | 21500 | 1.429 | 2.400 | 1.870 | 1.810 |
| 11017-12 | 10/15/98 | 3 | 5791 | 32000 | 2.357 | 4.760 | 2.570 | 3.890 |
| 11017-13 | 10/19/98 | 3 | 17896 | 61100 | 5.071 | 11.880 | 5.180 | 9.640 |
| 11017-14 | 10/22/98 | 9 | 41952 | 89300 | 7.857 | 17.700 | 11.700 | 15.380 |

| PANEL MEMBER | ORTHO VITROS HBsAg Reagent Pack | ABBOTT AxSYM HBsAg (V2) Reagent Pack | ABBOTT PRISM HBsAg | BIOTEST ELISA HBsAg | ABBOTT Auszyme MC Dynamic HBsAg | ABBOTT Auszyme Monoclonal HBsAg |
|--------------|--|---|--------------------------|---------------------------|--|--|
| | TEST DATE: 3/23/2000 | TEST DATE: 1/10/2000 | TEST DATE: UNK | TEST DATE: 4/18/2000 | TEST DATE: 12/15/1999 | TEST DATE: UNK |
| 11017-01 | 0.150 | 0.570 | 0.39 | 0.220 | 0.110 | 0.055 |
| 11017-02 | 0.190 | 0.600 | 0.36 | 0.240 | 0.290 | 0.073 |
| 11017-03 | 0.200 | 0.420 | 0.33 | 0.180 | 0.260 | 0.073 |
| 11017-04 | 0.140 | 0.470 | 0.39 | 0.140 | 0.110 | 0.109 |
| 11017-05 | 0.130 | 0.480 | 0.35 | 0.180 | 0.140 | 0.073 |
| 11017-06 | 0.120 | 0.430 | 0.40 | 0.140 | 0.140 | 0.127 |
| 11017-07 | 0.160 | 0.590 | 0.45 | 0.140 | 0.260 | 0.182 |
| 11017-08 | 0.190 | 0.530 | 0.53 | 0.220 | 0.400 | 0.145 |
| 11017-09 | 0.320 | 0.650 | 1.08 | 0.240 | 0.690 | 0.327 |
| 11017-10 | 0.540 | 0.820 | 2.62 | 0.290 | 0.800 | 0.673 |
| 11017-11 | 2.180 | 1.700 | 2.02 | 1.140 | 3.070 | 1.655 |
| 11017-12 | 5.500 | 2.920 | 14.85 | 1.460 | 5.250 | 4.727 |
| 11017-13 | 14.940 | 6.280 | 24.83 | 4.200 | 13.380 | 12.182 |
| 11017-14 | 27.520 | 12.330 | 46.94 | 4.970 | 18.230 | 14.909 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11017 Rev03
Effective Date: 06/15/2021

ZeptoMetrix®

HBV Seroconversion Panel Donor No. 68739 Part Number: HBV11017

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 4/18/2000 | ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK | ABBOTT HBeAg TEST DATE: 9/1/1999 | SORIN ETI-MAK 3 TEST DATE: 12/8/1999 | SORIN ETI-MAK 4 TEST DATE: UNK | ABBOTT ANTI-HBc TOTAL TEST DATE: 9/1/1999 |
|--------------|--|--|---|--|--|---|
| 11017-01 | 0.040 | 0.47 | NOT TESTED | 0.070 | 0.230 | NOT TESTED |
| 11017-02 | 0.190 | 0.47 | NOT TESTED | 0.070 | 0.250 | NOT TESTED |
| 11017-03 | 0.130 | 0.63 | NOT TESTED | 0.090 | 0.330 | NOT TESTED |
| 11017-04 | 0.160 | 0.48 | NOT TESTED | 0.090 | 0.300 | NOT TESTED |
| 11017-05 | 0.070 | 0.45 | NOT TESTED | 0.060 | 0.250 | NOT TESTED |
| 11017-06 | 0.130 | 0.50 | NOT TESTED | 0.060 | 0.200 | NOT TESTED |
| 11017-07 | 0.190 | 0.55 | NOT TESTED | 0.090 | 0.400 | NOT TESTED |
| 11017-08 | 0.310 | 0.59 | NOT TESTED | 0.130 | 0.300 | NOT TESTED |
| 11017-09 | 0.430 | 0.96/1.095 | NOT TESTED | 0.350 | 0.730 | NOT TESTED |
| 11017-10 | 0.810 | 1.31 | NOT TESTED | 0.630 | 1.030 | NOT TESTED |
| 11017-11 | 2.300 | 3.55 | NOT TESTED | 1.560 | 2.850 | NOT TESTED |
| 11017-12 | 4.810 | 7.07 | NOT TESTED | 3.500 | 5.700 | NOT TESTED |
| 11017-13 | 13.560 | 19.02 | NOT TESTED | 8.780 | 9.850 | NOT TESTED |
| 11017-14 | 23.130 | 25.80 | 0.405 | 11.910 | 25.570 | 0.464 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11017 Rev03
Effective Date: 06/15/2021