

ZeptoMetrix®

HBV Seroconversion Panel Donor No. 70970 Part Number: HBV11026

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 9/16/1999	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	ROCHE HBV Monitor Assay (copies/mL) TEST DATE: UNK	GENETIC SYSTEMS HBsAg 2.0 Static TEST DATE: UNK	ROCHE HBV Ampliscreen TEST DATE: UNK	VENDOR HBsAg TEST DATE: UNK	ORTHO VITROS Eci HBsAg TEST DATE: 8/30/2018
11026-01	02/26/99	5	NOT DETECTED	200	NEGATIVE	POSITIVE/NEGATIVE	NEGATIVE	0.06
11026-02	03/02/99	7	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	NEGATIVE	0.07
11026-03	03/05/99	7	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	NEGATIVE	0.07
11026-04	03/09/99	11	NOT DETECTED	<100	NEGATIVE	POSITIVE	NEGATIVE	0.17
11026-05	03/12/99	12	NOT DETECTED	<100	NEGATIVE	POSITIVE	NEGATIVE	0.06
11026-06	03/16/99	13	NOT DETECTED	<100	NEGATIVE	POSITIVE	NEGATIVE	0.07
11026-07	03/19/99	20	NOT DETECTED	<100	NEGATIVE	POSITIVE	NEGATIVE	0.06
11026-08	03/23/99	13	<52	200	NEGATIVE	POSITIVE	NEGATIVE	0.06
11026-09	03/26/99	11	78	200	NEGATIVE	POSITIVE	NEGATIVE	0.06
11026-10	03/30/99	10	139	200	NEGATIVE	POSITIVE	NEGATIVE	0.08
11026-11	04/03/99	12	333	3,100	NEGATIVE	POSITIVE	NEGATIVE	0.28
11026-12	04/06/99	5	1210	5,200	NEGATIVE	POSITIVE	NEGATIVE	0.31
11026-13	04/10/99	10	3915	10,000	NEGATIVE	POSITIVE	NEGATIVE	1.54
11026-14	04/13/99	5	9835	32,000	POSITIVE	POSITIVE	POSITIVE	4.76
11026-15	04/17/99	14	60639	170,000	POSITIVE	POSITIVE	POSITIVE	19.0
11026-16	04/20/99	6	148852	350,000	POSITIVE	POSITIVE	POSITIVE	51.2

PANEL MEMBER	ABBOTT PRISM HBsAg TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ORTHO HBsAg Proc B TEST DATE: UNK	ORTHO VITROS Eci ANTI-HBc TOTAL TEST DATE: 8/30/2018	ORTHO VITROS Eci HBeAg TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS Eci ANTI-HBs QUANT TEST DATE: 8/30/2018
11026-01	0.66	0.49	0.067	3.09	0.13	0.17	0.02	0.0
11026-02	0.82	0.49	0.067	3.26	0.15	0.15	0.02	0.0
11026-03	0.46	0.50	0.067	3.44	0.14	0.18	0.02	0.0
11026-04	0.86	0.45	0.067	3.61	0.13	0.17	0.02	0.0
11026-05	0.47	0.46	0.067	3.39	0.13	0.16	0.02	0.0
11026-06	0.90	0.51	0.067	3.35	0.14	0.16	0.02	0.0
11026-07	0.45	0.58	0.033	3.15	0.13	0.15	0.02	0.0
11026-08	0.99	0.59	0.067	3.19	0.12	0.17	0.02	0.0
11026-09	0.80	0.50	0.033	3.34	0.13	0.16	0.02	0.0
11026-10	0.83	0.55	0.033	3.23	0.14	0.17	0.02	0.0
11026-11	1.01	0.68	0.033	3.58	0.13	0.18	0.02	0.0
11026-12	2.68	1.09	0.200	3.32	0.13	0.17	0.02	0.0
11026-13	4.62	2.80	0.600	3.04	0.13	0.17	0.03	0.0
11026-14	10.66	5.86	1.567	3.12	0.17	0.16	0.03	0.0
11026-15	6.71	22.73	6.600	3.15	0.40	0.16	0.02	0.0
11026-16	91.12	32.07	16.533	3.55	1.67	0.16	0.03	0.0

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11026 Rev04
Effective Date: 08/12/2021