

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No. 71840 Part Number: HBV11028

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 7/1/1999	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE HBV Monitor Assay (copies/mL)  TEST DATE: UNK	ROCHE HBV Ampliscreen  TEST DATE: UNK	VENDOR HBsAg  TEST DATE: UNK	ABBOTT PRISM HBsAg  TEST DATE: UNK
11028-01	05/01/99	16	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	0.40
11028-02	05/03/99	18	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	0.36
11028-03	05/13/99	21	231	200	POSITIVE	NEGATIVE	0.51
11028-04	05/15/99	21	105	200	POSITIVE	NEGATIVE	0.47
11028-05	05/23/99	20	357	500	POSITIVE	NEGATIVE	1.10
11028-06	05/28/99	16	1296	8600	POSITIVE	NEGATIVE	4.04
11028-07	05/31/99	10	2152	9800	POSITIVE	NEGATIVE	3.38
11028-08	06/05/99	22	11555	22000	POSITIVE	NEGATIVE	6.48
11028-09	06/10/99	20	37390	160000	POSITIVE	NEGATIVE	15.53
11028-10	06/16/99	27	7290	49000	POSITIVE	POSITIVE	18.52

PANEL MEMBER	ORTHO HBsAg Proc B Rpt  TEST DATE: UNK	ABBOTT HBsAg Proc C S/CO  TEST DATE: UNK	ABBOTT HBsAg Proc B S/CO  TEST DATE: UNK	ORTHO HBsAg Proc B  TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK	GENETIC SYSTEMS HBsAg 2.0 Static  TEST DATE: UNK
11028-01	NOT TESTED	0.000	0.019	0.067	1.01/1.10	NEGATIVE
11028-02	NOT TESTED	0.138	0.151	0.033	0.95/1.02	NEGATIVE
11028-03	NOT TESTED	0.276	0.094	0.067	0.94/0.92	NEGATIVE
11028-04	NOT TESTED	0.103	0.113	0.067	0.91/0.92	NEGATIVE
11028-05	NOT TESTED	0.345	0.283	0.133	1.11/1.13	NEGATIVE
11028-06	NOT TESTED	0.483	0.830	0.300	1.87/1.73	NEGATIVE
11028-07	NOT TESTED	0.793	1.019	0.467	2.12	NEGATIVE
11028-08	NOT TESTED	1.345	1.981	0.533	3.18	NEGATIVE
11028-09	1.500	3.138	4.774	1.367	8.50	NEGATIVE
11028-10	NOT TESTED	4.517	6.396	2.300	10.36	POSITIVE

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11028 Rev04  
Effective Date: 08/13/2021



**HBV Seroconversion Panel Donor No. 71840**  
**Part Number: HBV11028**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECI HBsAg	ORTHO VITROS ECI ANTI-HBc TOTAL	ORTHO VITROS ECI HBeAg	ORTHO VITROS ECI ANTI-HBe	ORTHO VITROS ECI ANTI-HBc IgM	ORTHO VITROS ECI ANTI-HBs QUANT
	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11028-01	0.08	3.47	0.14	0.19	0.06	0.0
11028-02	0.22	3.36	0.16	0.16	0.04	0.6
11028-03	0.19	3.18	0.15	0.16	0.04	0.0
11028-04	0.17	3.55	0.14	0.17	0.05	0.6
11028-05	0.23	3.47	0.20	0.18	0.05	0.0
11028-06	0.82	3.47	0.15	0.19	0.05	0.0
11028-07	1.06	4.00	0.16	0.16	0.04	0.0
11028-08	2.96	3.62	0.17	0.19	0.05	0.0
11028-09	8.31	3.53	0.23	0.16	0.04	0.0
11028-10	9.49	3.43	0.26	0.17	0.04	0.0

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11028 Rev04  
Effective Date: 08/13/2021