



HBV Seroconversion Panel Donor No. 70292
Part Number: HBV11031

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT	ROCHE	ROCHE	ORTHO
		Normal Range (10-45u/L)	HBV REAL TIME CP/ML	HBV Monitor Assay (copies/mL)	HBV Ampliscreen	HBsAg Proc B
		TEST DATE: 3/8/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11031-01	12/27/98	3	NOT DETECTED	NOT DETECTED	NEGATIVE	0.100
11031-02	12/31/98	0	NOT DETECTED	NOT DETECTED	NEGATIVE	0.100
11031-03	01/03/99	1	NOT DETECTED	NOT DETECTED	NEGATIVE	0.033
11031-04	01/07/99	0	NOT DETECTED	100	POSITIVE	0.067
11031-05	01/10/99	0	<52	NOT DETECTED	NEGATIVE	0.067
11031-06	01/14/99	4	NOT DETECTED	NOT DETECTED	NEGATIVE	0.067
11031-07	01/17/99	0	<52	200	POSITIVE	0.100
11031-08	01/21/99	0	<52	200	POSITIVE	0.067
11031-09	01/24/99	0	90	200	POSITIVE	0.100
11031-10	01/31/99	3	220	900	POSITIVE	0.200
11031-11	02/04/99	4	1129	2800	POSITIVE	0.600
11031-12	02/07/99	7	2414	3500	POSITIVE	1.167
11031-13	02/11/99	NOT TESTED	11824	24000	POSITIVE	3.867
11031-14	02/14/99	3	10783	57000	POSITIVE	8.767
11031-15	02/18/99	4	43929	100000	POSITIVE	25.23
11031-16	02/21/99	3	50438	370000	POSITIVE	69.4

PANEL MEMBER	ABBOTT	ORTHO	ABBOTT	ABBOTT	GENETIC SYSTEMS	ABBOTT
	HBsAg Proc B	HBsAg Proc B Rot	HBsAg Proc C	PRISM HBsAg	HBsAg 2.0 Static	MUREX HBsAg Version 3
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11031-01	0.052	NOT TESTED	0.207	1.36	NEGATIVE	0.46
11031-02	0.052	NOT TESTED	0.103	1.02	NEGATIVE	0.46
11031-03	0.155	NOT TESTED	0.103	0.81	NEGATIVE	0.52
11031-04	0.103	NOT TESTED	0.207	0.72	NEGATIVE	0.49
11031-05	0.052	NOT TESTED	0.138	0.62	NEGATIVE	0.51
11031-06	0.241	NOT TESTED	0.172	0.78	NEGATIVE	0.51
11031-07	1.776	NOT TESTED	0.034	0.75	NEGATIVE	0.49
11031-08	1.793	NOT TESTED	0.069	0.64	NEGATIVE	0.49
11031-09	0.138	NOT TESTED	0.172	1.27	NEGATIVE	0.54
11031-10	0.328	NOT TESTED	0.448	1.91	NEGATIVE	1.05
11031-11	0.569	NOT TESTED	0.759	4.26	NEGATIVE	2.27
11031-12	1.931	1	1.207	7.12	NEGATIVE	4.04
11031-13	6.069	NOT TESTED	4.483	23.00	NOT TESTED	12.37
11031-14	10.793	NOT TESTED	11.586	54.82	POSITIVE	24.86
11031-15	>34.483	NOT TESTED	29.310	137.42	POSITIVE	30.57
11031-16	>34.483	NOT TESTED	66.414	277.56	POSITIVE	30.52

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

ZeptoMetrix®

HBV Seroconversion Panel Donor No. 70292 Part Number: HBV11031

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi HBsAg	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBs QUANT
	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11031-01	0.07	2.91	0.13	0.21	0.02	0.0
11031-02	0.05	3.21	0.14	0.17	0.01	0.0
11031-03	0.05	3.52	0.13	0.21	0.01	0.0
11031-04	0.05	3.28	0.13	0.19	0.01	0.0
11031-05	0.05	3.27	0.14	0.18	0.01	0.1
11031-06	0.05	3.39	0.14	0.16	0.01	0.0
11031-07	0.05	3.10	0.13	0.21	0.01	0.8
11031-08	0.12	3.07	0.13	0.21	0.01	0.3
11031-09	0.11	3.07	0.13	0.21	0.02	0.1
11031-10	0.27	3.13	0.14	0.19	0.01	0.4
11031-11	0.85	3.16	0.13	0.17	0.01	0.4
11031-12	2.72	3.02	0.16	0.20	0.01	0.2
11031-13	12.2	3.07	1.06	0.21	0.02	0.3
11031-14	29.3	3.21	0.16	0.21	0.01	0.1
11031-15	88.3	3.10	0.27	0.19	0.01	0.0
11031-16	225	3.16	0.87	0.18	0.01	0.1

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11031 Rev05
Effective Date: 08/13/2021