



**HBV Seroconversion Panel Donor No. 72474  
Part Number: HBV11056**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 8/17/1999	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE HBV Monitor Assay (copies/mL)  TEST DATE: UNK	ROCHE HBV Ampliscreen  TEST DATE: UNK	GENETIC SYSTEMS HBsAg 2.0 Static  TEST DATE: UNK	ORTHO HBsAg Proc B Rpt  TEST DATE: UNK
11056-01	06/15/99	5	NOT DETECTED	Not Tested	Negative	Negative	Not Tested
11056-02	06/20/99	5	NOT DETECTED	Not Tested	Negative	Negative	Not Tested
11056-03	06/25/99	1	NOT DETECTED	Not Tested	Negative	Negative	Not Tested
11056-04	07/02/99	6	NOT DETECTED	Not Tested	Negative	Negative	Not Tested
11056-05	07/04/99	6	NOT DETECTED	200	Positive	Negative	Not Tested
11056-06	07/10/99	7	124	200	Positive	Negative	Not Tested
11056-07	07/12/99	5	105	200	Positive	Negative	Not Tested
11056-08	07/18/99	4	680	8,500	Positive	Negative	Not Tested
11056-09	07/20/99	0	1296	9,200	Positive	Negative	Not Tested
11056-10	08/02/99	5	579	6,700	Positive	Positive	1.133
11056-11	08/04/99	2	1155	500	Positive	Positive	1.633

PANEL MEMBER	ORTHO HBsAg Proc B  TEST DATE: UNK	ABBOTT HBsAg Proc B  TEST DATE: UNK	ABBOTT HBsAg Proc C  TEST DATE: UNK	ABBOTT PRISM HBsAg  TEST DATE: UNK	ORTHO VITROS ECi HBsAg  TEST DATE: 8/16/2010	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL  TEST DATE: 8/16/2010	ORTHO VITROS ECi ANTI-HBs QUANT  TEST DATE: 8/16/2010
11056-01	0.065	0.057	0.000	0.57	0.14	0.47	2.69	0.70
11056-02	0.097	0.019	0.111	0.30	0.13	0.46	2.70	0.30
11056-03	0.065	0.019	0.037	0.43	0.16	0.46	2.65	0.20
11056-04	0.032	0.019	0.222	0.71	0.16	0.48	2.86	0.40
11056-05	0.129	0.057	0.148	0.77	0.15	0.45	3.01	0.90
11056-06	0.161	0.038	0.185	0.89	0.18	0.50	2.95	0.70
11056-07	0.097	0.075	0.074	0.79	0.18	0.53	3.03	0.00
11056-08	0.161	0.189	0.148	1.97	0.51	0.86	2.69	0.40
11056-09	0.226	0.396	0.296	2.44	0.56	1.09	3.01	0.50
11056-10	1.129	2.962	1.704	10.81	5.77	4.73	3.00	0.80
11056-11	1.387	2.849	2.704	12.62	8.89	5.32	2.69	0.10

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**  
ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)