



**HBV Seroconversion Panel Donor No. 72324
Part Number: HBV11059**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	ROCHE HBV Monitor Assay (copies/mL)	ROCHE HBV Ampliscreen	GENETIC SYSTEMS HBsAg 2.0 Static	VENDOR HBsAg
		TEST DATE: 8/5/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11059-01	06/19/99	12	NOT DETECTED	<100	POSITIVE	NEGATIVE	NEGATIVE
11059-02	06/21/99	10	<52	200	POSITIVE	NEGATIVE	NEGATIVE
11059-03	06/29/99	9	52	200	POSITIVE	NEGATIVE	NEGATIVE
11059-04	07/06/99	6	493	500	POSITIVE	NEGATIVE	NEGATIVE
11059-05	07/16/99	9	4195	17,000	POSITIVE	NEGATIVE	NEGATIVE
11059-06	07/21/99	2	13892	77,000	POSITIVE	POSITIVE	POSITIVE
11059-07	07/23/99	11	30392	160,000	POSITIVE	POSITIVE	POSITIVE
11059-08	07/29/99	16	178959	700,000	POSITIVE	POSITIVE	POSITIVE
11059-09	07/31/99	4	247033	800,000	POSITIVE	POSITIVE	POSITIVE

PANEL MEMBER	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	ABBOTT HBsAg Proc C	ABBOTT HBsAg Proc B	ORTHO HBsAg Proc B	ORTHO HBsAg Proc B Rpt
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11059-01	0.39	0.57	0.296	0.019	0.000	NOT TESTED
11059-02	0.43	0.63	0.111	0.019	0.031	NOT TESTED
11059-03	0.41	0.63	0.148	0.057	0.031	NOT TESTED
11059-04	0.96	0.96	0.370	0.264	0.156	NOT TESTED
11059-05	0.16	5.76	2.111	3.245	1.750	1.633
11059-06	40.76	14.12	6.370	10.057	4.969	NOT TESTED
11059-07	13.14	23.67	10.704	15.755	8.750	NOT TESTED
11059-08	44.68	29.30	35.889	>37.736	37.188	NOT TESTED
11059-09	216.88	29.17	51.556	>37.736	53.156	NOT TESTED

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



HBV Seroconversion Panel Donor No. 72324
Part Number: HBV11059

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS Eci HBsAg	ORTHO VITROS Eci ANTI-HBc TOTAL	ORTHO VITROS Eci HBeAg	ORTHO VITROS Eci ANTI-HBe	ORTHO VITROS Eci ANTI-HBc IgM	ORTHO VITROS Eci ANTI-HBs QUANT
	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11059-01	0.07	2.87	0.14	0.03	0.03	0.1
11059-02	0.05	2.67	0.13	0.16	0.02	0.0
11059-03	0.08	2.76	0.13	0.19	0.02	0.0
11059-04	0.19	2.90	0.14	0.16	0.02	0.0
11059-05	5.40	2.64	0.18	0.17	0.02	0.0
11059-06	16.8	2.79	0.19	0.16	0.02	1.0
11059-07	30.1	2.91	0.19	0.16	0.02	0.0
11059-08	137	2.95	0.61	0.16	0.02	0.0
11059-09	201	2.85	0.89	0.16	0.02	0.0

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11059 Rev04
Effective Date:08/31/2021