



HBV Seroconversion Panel Donor No. 71922
Part Number: HBV11062

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ALT Normal Range (10-45u/L) | ABBOTT HBV REAL TIME CP/ML | ROCHE HBV Monitor Assay (copies/mL) | ROCHE HBV Ampliscreen | GENETIC SYSTEMS HBsAg 2.0 Static | VENDOR HBsAg | ABBOTT MUREX HBsAg Version 3 | ABBOTT PRISM HBsAg |
|--------------|--------------|-----------------------------|----------------------------|-------------------------------------|-----------------------|----------------------------------|----------------|------------------------------|--------------------|
| | | TEST DATE: 8/6/1999 | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK |
| 11062-01 | 05/18/99 | 6 | NOT DETECTED | NOT TESTED | NEGATIVE | NEGATIVE | NEGATIVE | 0.44 | 1.04 |
| 11062-02 | 05/20/99 | 8 | NOT DETECTED | NOT TESTED | NEGATIVE | NEGATIVE | NEGATIVE | 0.41 | 1.77 |
| 11062-03 | 05/25/99 | 7 | <52 | 200 | POSITIVE | NEGATIVE | NEGATIVE | 0.47 | 0.98 |
| 11062-04 | 05/27/99 | 0 | NOT DETECTED | 200 | POSITIVE | NEGATIVE | NEGATIVE | 0.45 | 1.08 |
| 11062-05 | 06/01/99 | 5 | 110 | 200 | POSITIVE | NEGATIVE | NEGATIVE | 0.69 | 0.92 |
| 11062-06 | 06/04/99 | 9 | 130 | 200 | POSITIVE | NEGATIVE | NEGATIVE | 0.92 | 1.82 |
| 11062-07 | 06/09/99 | 8 | 665 | 8000 | POSITIVE | NEGATIVE | NEGATIVE | 1.91 | 3.06 |
| 11062-08 | 06/11/99 | 14 | 948 | 9900 | POSITIVE | POSITIVE | POSITIVE | 2.75 | 5.09 |
| 11062-09 | 06/16/99 | 7 | 8969 | 75000 | POSITIVE | POSITIVE | POSITIVE | 8.49 | 21.33 |
| 11062-10 | 06/18/99 | 4 | 12381 | 110000 | POSITIVE | POSITIVE | POSITIVE | 15.21 | 27.16 |
| 11062-11 | 06/23/99 | 6 | 85655 | 650000 | POSITIVE | POSITIVE | POSITIVE | 30.04 | 112.40 |
| 11062-12 | 06/26/99 | 3 | 220168 | 1600000 | POSITIVE | POSITIVE | POSITIVE | 30.59 | 180.17 |

| PANEL MEMBER | ABBOTT HBsAg Proc C | ABBOTT HBsAg Proc B | ORTHO HBsAg Proc B | ORTHO VITROS ECi HBsAg | ORTHO VITROS ECi ANTI-HBc TOTAL | ORTHO VITROS ECi HBeAg | ORTHO VITROS ECi ANTI-HBe | ORTHO VITROS ECi ANTI-HBc IgM | ORTHO VITROS ECi ANTI-HBs QUANT |
|--------------|---------------------|---------------------|--------------------|------------------------|---------------------------------|------------------------|---------------------------|-------------------------------|---------------------------------|
| | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 8/30/2018 | TEST DATE: 8/30/2018 | TEST DATE: 8/30/2018 | TEST DATE: 8/30/2018 | TEST DATE: 8/30/2018 | TEST DATE: 8/30/2018 |
| 11062-01 | 0.185 | 0.019 | 0.031 | 0.05 | 3.33 | 0.11 | 0.17 | 0.01 | 0.0 |
| 11062-02 | 0.222 | 0.019 | 0.031 | 0.05 | 3.20 | 0.12 | 0.17 | 0.01 | 0.0 |
| 11062-03 | 0.037 | 0.019 | 0.094 | 0.05 | 3.24 | 0.11 | 0.13 | 0.01 | 0.0 |
| 11062-04 | 0.148 | 0.302 | 0.000 | 0.05 | 3.19 | 0.12 | 0.14 | 0.01 | 0.0 |
| 11062-05 | 0.259 | 0.038 | 0.094 | 0.08 | 3.39 | 0.12 | 0.15 | 0.01 | 0.0 |
| 11062-06 | 0.296 | 0.340 | 0.281 | 0.24 | 3.12 | 0.12 | 0.16 | 0.01 | 0.0 |
| 11062-07 | 0.778 | 1.226 | 0.594 | 1.48 | 3.28 | 0.12 | 0.16 | 0.01 | 0.0 |
| 11062-08 | 1.114 | 2.113 | 1.188 | NOT TESTED | NOT TESTED | NOT TESTED | NOT TESTED | NOT TESTED | NOT TESTED |
| 11062-09 | 5.222 | 7.642 | 5.281 | 14.8 | 3.45 | 0.22 | 0.15 | 0.01 | 0.0 |
| 11062-10 | 7.667 | 15.226 | 7.656 | 26.3 | 3.33 | 0.38 | 0.15 | 0.01 | 0.0 |
| 11062-11 | 42.852 | >37.736 | 44.063 | 136 | 3.13 | 4.49 | 0.13 | 0.01 | 0.0 |
| 11062-12 | 76.259 | >37.736 | 88.344 | 364 | 3.20 | 20.8 | 0.09 | 0.01 | 0.0 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com