



HBV Seroconversion Panel Donor No. 71782
Part Number: HBV11064

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	ROCHE HBV Monitor Assay (copies/mL)	ROCHE HBV Ampliscreen	GENETIC SYSTEMS HBsAg 2.0 Static	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3
		TEST DATE: 8/24/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11064-01	04/05/99	14	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	0.58	0.50
11064-02	04/07/99	4	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	0.50	0.48
11064-03	04/17/99	8	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	0.86	0.50
11064-04	04/20/99	3	<52	NOT TESTED	NEGATIVE	NEGATIVE	0.56	0.49
11064-05	04/23/99	7	NOT DETECTED	100	POSITIVE	NEGATIVE	0.44	0.48
11064-06	04/27/99	3	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	0.60	0.49
11064-07	04/30/99	10	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	0.45	0.46
11064-08	06/12/99	11	13266	16000	POSITIVE	POSITIVE	23.43	8.93
11064-09	06/16/99	12	37390	110000	POSITIVE	POSITIVE	50.53	25.57

PANEL MEMBER	ABBOTT HBsAg Proc C	ABBOTT HBsAg Proc B	ORTHO HBsAg Proc B	ORTHO VITROS ECi HBsAg	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBs QUANT
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11064-01	0.111	0.019	0.063	0.22	3.51	0.15	0.22	0.03	0.0
11064-02	0.037	0.208	0.031	0.07	3.21	0.17	QNS	QNS	QNS
11064-03	0.370	0.132	0.094	0.07	3.22	0.14	0.24	0.04	0.0
11064-04	0.185	0.075	0.063	0.10	3.51	0.19	0.20	0.03	0.1
11064-05	0.296	0.038	0.063	0.08	3.32	0.17	0.20	0.03	0.1
11064-06	0.148	0.038	0.031	0.10	3.15	0.16	0.20	0.03	0.0
11064-07	0.000	0.057	0.188	0.08	3.56	0.16	0.19	0.02	0.6
11064-08	3.481	6.472	3.438	12.6	3.73	0.20	0.21	0.03	0.0
11064-09	10.074	15.547	10.250	33.2	3.30	0.26	0.20	0.03	0.0

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11064 Rev04
 Effective Date: 08/31/2021