



**HBV Seroconversion Panel Donor No. 73299**  
**Part Number: HBV11069**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT	ROCHE	ROCHE	GENETIC	GENETIC
		Normal Range (10-45u/L)	HBV REAL TIME CP/ML	HBV Monitor Assay (copies/mL)	HBV Ampliscreen	SYSTEMS HBsAg 2.0 Static	SYSTEMS HBsAg
		TEST DATE: 10/18/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11069-01	07/10/99	9	NOT DETECTED	NOT DETECTED	NEGATIVE	NEGATIVE	0.394
11069-02	07/12/99	5	NOT DETECTED	NOT DETECTED	NEGATIVE	NEGATIVE	0.472
11069-03	07/28/99	10	NOT DETECTED	NOT DETECTED	NEGATIVE	NEGATIVE	0.551
11069-04	08/11/99	16	NOT DETECTED	NOT DETECTED	NEGATIVE	NEGATIVE	0.315
11069-05	08/23/99	10	<52	200	POSITIVE	NEGATIVE	0.315
11069-06	08/25/99	11	76	200	POSITIVE	NEGATIVE	0.315
11069-07	08/30/99	15	145	200	POSITIVE	NEGATIVE	0.394
11069-08	09/01/99	13	196	4700	POSITIVE	NEGATIVE	0.394
11069-09	09/06/99	11	553	2600	POSITIVE	NEGATIVE	0.472
11069-10	09/08/99	8	357	1100	POSITIVE	NEGATIVE	0.709
11069-11	09/13/99	11	696	7600	POSITIVE	NEGATIVE	0.866
11069-12	09/15/99	9	1267	7600	POSITIVE	NEGATIVE	0.709
11069-13	09/29/99	19	7124	35000	POSITIVE	POSITIVE	3.228
11069-14	10/02/99	13	9178	48000	POSITIVE	POSITIVE	4.252
11069-15	10/06/99	10	9835	36000	POSITIVE	POSITIVE	7.323

PANEL MEMBER	SORIN	ABBOTT	ABBOTT	ORTHO	ABBOTT	ABBOTT	SORIN
	HBsAg	PRISM HBsAg	MUREX HBsAg Version 3	HBsAg Proc B Rpt	HBsAg Proc B	HBsAg Proc C	HBeAg
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/12/2000	TEST DATE: UNK	TEST DATE: UNK
11069-01	0.220	0.73	0.58	NOT TESTED	0.019	0.222	0.246
11069-02	0.283	0.77	0.58	NOT TESTED	0.038	0.148	0.130
11069-03	0.157	0.57	0.58	NOT TESTED	0.096	0.185	0.072
11069-04	0.220	0.59	0.63	NOT TESTED	0.058	0.148	0.174
11069-05	0.157	0.40	0.62	NOT TESTED	0.096	0.148	0.174
11069-06	0.220	0.56	0.65	NOT TESTED	0.038	0.185	0.174
11069-07	0.157	0.82	0.78	NOT TESTED	0.019	0.111	0.159
11069-08	0.220	0.92	0.76	NOT TESTED	0.096	0.148	0.188
11069-09	0.346	1.65	1.13	NOT TESTED	0.212	0.333	0.130
11069-10	0.377	2.26	1.45	NOT TESTED	0.365	0.407	0.087
11069-11	0.346	2.87	1.75	NOT TESTED	0.538	0.519	0.130
11069-12	0.283	3.12	1.95	NOT TESTED	0.635	0.667	0.145
11069-13	0.314	18.03	9.11	2.200	3.519	3.037	0.145
11069-14	0.314	27.94	16.47	NOT TESTED	5.923	4.667	0.159
11069-15	0.503	42.67	23.95	NOT TESTED	10.577	8.000	0.174

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11069 Rev04  
 Effective Date: 08/31/2021



**HBV Seroconversion Panel Donor No. 73299**  
**Part Number: HBV11069**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SORIN HBeAg (Co/S)	ORTHO VITROS Eci HBsAg	ORTHO VITROS Eci ANTI-HBc TOTAL	ORTHO VITROS Eci HBeAg	ORTHO VITROS Eci ANTI-HBe	ORTHO VITROS Eci ANTI-HBe IgM	ORTHO VITROS Eci ANTI-HBs QUANT
	TEST DATE: UNK	TEST DATE: 8/20/2018	TEST DATE: 8/20/2018	TEST DATE: 8/20/2018	TEST DATE: 8/20/2018	TEST DATE: 8/20/2018	TEST DATE: 8/20/2018
11069-01	0.50	0.08	2.57	0.13	0.19	0.03	0.0
11069-02	0.50	0.07	2.71	0.14	0.17	0.03	0.6
11069-03	0.50	0.06	2.70	0.13	0.16	0.02	0.2
11069-04	0.50	0.08	2.78	0.13	0.16	0.02	0.0
11069-05	0.50	0.06	2.89	0.15	0.16	0.02	0.0
11069-06	0.52	0.12	2.71	0.14	0.19	0.02	0.0
11069-07	0.52	0.08	2.58	0.15	0.19	0.02	0.0
11069-08	0.53	0.11	2.66	0.15	NOT TESTED	0.02	0.0
11069-09	0.50	0.53	2.43	0.15	NOT TESTED	0.02	0.0
11069-10	0.50	0.32	2.34	0.14	NOT TESTED	NOT TESTED	NOT TESTED
11069-11	0.49	0.74	2.27	0.15	0.18	NOT TESTED	NOT TESTED
11069-12	0.52	0.82	2.27	0.14	0.20	0.03	0.0
11069-13	0.51	8.47	2.04	0.17	0.20	0.03	0.0
11069-14	0.55	14.4	2.22	0.21	0.19	0.03	0.0
11069-15	0.52	23.7	1.90	0.32	0.22	0.03	0.0

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)