



## HBV Seroconversion Panel Donor No.61248 Part Number: HBV6271

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SORIN ETI-MAK 3 HBsAg TEST DATE: 1/16/1997	ABBOTT PRISM HBsAg TEST DATE: 2/5/1997	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK	ORTHO AMERLITE HBsAg II TEST DATE: 7/14/1997	ABBOTT AxSYM HBsAg TEST DATE: 1/6/1997	ABBOTT Austria II-125 TEST DATE: 12/3/1996	ABBOTT ANTI-HBe (EIA) TEST DATE: 12/17/1996	ABBOTT ANTI-HBc TOTAL (EIA) TEST DATE: 12/17/1996	ABBOTT Auszyme MC Dynamic TEST DATE: 1/21/1997
6271-01	0.35	0.39	0.57	0.09	0.50	0.32	0.526	0.426	0.45
6271-02	0.43	0.70	0.80	0.15	0.64	0.38	0.560	0.448	0.64
6271-03	0.82	1.81	1.69	0.39	0.91	0.51	0.593	0.451	1.70
6271-04	3.38	9.49	9.06	3.98	3.11	1.67	0.556	0.429	9.85
6271-05	19.38	56.7	32.61	38.55	19.24	12.67	0.549	0.417	47.61

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 11/15/1996	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	BIOMERIEUX VIDAS HBsAg TEST DATE: 1/16/1997	BOEHR. MANNHEIM ELECSYS HBsAg TEST DATE: 10/9/1997	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 4/28/1998	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 7/2/1997	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	DIASORIN HBc IgM TEST DATE: 10/5/2011
6271-01	03/16/1996	9	215	0.16	0.53	0.57	0.20	119	0.21
6271-02	03/19/1996	14	763	0.25	0.72	0.49	0.43	2150	0.17
6271-03	03/23/1996	11	2359	0.69	1.43	0.60	1.36	8660	0.17
6271-04	03/28/1996	10	19623	3.96	7.11	1.35	8.67	78100	0.17
6271-05	04/03/1996	13	167015	32.68	50.94	7.59	58.70	372000	0.17

PANEL MEMBER	MUREX HBsAg TEST DATE: 1/9/1997	ABBOTT Auszyme Monoclonal TEST DATE: 9/30/1997	ABBOTT HBsAg (EIA) TEST DATE: 11/15/1996	ABBOTT HBeAg (EIA) TEST DATE: 12/17/1996	ABBOTT IMx HBsAg Reagent Pack TEST DATE: 3/13/1997	BIOTEST ELISA HBsAg TEST DATE: 3/3/1997	SANOFI Monolisa HBsAg 2 <sup>nd</sup> Generation TEST DATE: 3/6/1997	ROCHE Cobas Core HBsAg II (EIA) TEST DATE: 12/17/1996
6271-01	0.68	0.19	0.158	0.339	0.47	0.20	0.49	0.28
6271-02	0.60	0.28	0.263	0.339	0.53	0.28	0.46	0.47
6271-03	0.73	0.82	0.895	0.421	0.63	0.67	0.89	0.95
6271-04	1.95	5.81	6.491	0.314	2.01	4.18	2.40	6.10
6271-05	10.35	36.35	>35.088	0.438	9.50	23.76	14.99	41.99

PANEL MEMBER	BEHRINGW ERKE Enzygnost HBsAg Monoclonal II TEST DATE: 12/19/1996	ORTHO VITROS ECi HBsAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/30/2018	ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/30/2018
6271-01	0.49	0.07	3.13	0.12	0.19	0.01	0.0
6271-02	0.46	0.13	3.30	0.13	0.19	0.01	0.5
6271-03	0.69	0.60	3.07	0.12	0.19	0.01	0.0
6271-04	2.40	6.62	3.31	0.15	0.19	0.01	0.0
6271-05	14.99	60.3	3.24	0.37	0.18	0.01	0.0

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti-HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6271 Rev03  
Effective Date: 06/23/2021