



HBV Seroconversion Panel Donor No.61799
Part Number: HBV6274

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	BIOMERIEUX VIDAS HBsAg	BOEHR. MANNHEIM ELECSYS HBsAg	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	SORIN ETI- MAK 3 HBsAg	ABBOTT HBsAg (EIA)	ABBOTT PRISM HBsAg
		TEST DATE: 11/18/1996	TEST DATE: UNK	TEST DATE: 1/16/1997	TEST DATE: 10/9/1997	TEST DATE: 12/18/1996	TEST DATE: 1/16/1997	TEST DATE: 11/19/1996	TEST DATE: 8/27/2009
6274-01	07/03/96	1	857	0.48	1.03	0.51	0.63	0.617	2.01
6274-02	07/07/96	8	1790	0.72	1.65	0.68	1.31	1.267	3.41
6274-03	07/11/96	12	4929	1.24	3.4	0.81	2.34	3.167	7.15
6274-04	07/14/96	5	13892	1.69	5.33	1.07	4.02	5.783	11.78
6274-05	07/18/96	9	32565	7.41	18.51	3.38	14.4	15.583	42.03
6274-06	07/21/96	13	44953	18.9	45.44	7.37	31.7	>33.333	85.24
6274-07	07/25/96	7	215156	57.25	168.30	19.76	41.18	>33.333	184.16

PANEL MEMBER	ABBOTT MUREX HBsAg	ABBOTT MUREX HBsAg Version 3	ABBOTT AxSYM HBsAg	BIOTEST ELISA HBsAg	ABBOTT IMx HBsAg Reagent Pack	SANOFI Monolisa HBsAg 2 nd Generation	ROCHE Cobas Core HBsAg II (EIA)	ABBOTT Auszyme MC Dynamic	ABBOTT Auszyme Monoclonal
	TEST DATE: 1/9/1997	TEST DATE: 8/27/2009	TEST DATE: 1/6/1997	TEST DATE: 3/3/1997	TEST DATE: 3/13/1997	TEST DATE: 3/6/1997	TEST DATE: 12/17/1996	TEST DATE: 1/21/1997	TEST DATE: 9/30/1997
6274-01	0.71	1.06	0.77	0.34	0.59	0.56	0.68	0.97	0.48
6274-02	0.85	1.87	1.06	0.63	0.86	0.82	1.29	2.24	1.23
6272-03	1.05	3.67	2.15	1.07	1.49	1.22	2.24	5.24	3.07
6272-04	1.64	6.35	2.95	2.06	2.17	1.75	3.09	8.48	6.22
6272-05	4.09	20.64	9.45	9.86	6.17	7.64	17.0	28.94	17.98
6272-06	8.57	32.59	20.2	21.35	11.29	18.45	43.7	OVER	OVER
6272-07	19.4	34.16	45.66	43.19	20.13	51.25	74.07	OVER	OVER

PANEL MEMBER	BEHRING WERKE Enzygnost HBsAg Monoclonal II	ABBOTT Austria II-125	ORTHO AMERLITE HBsAg II	ORTHO HBsAg ELISA Testsystem 3	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL (EIA) OD Negative > 0.586 Positive ≤ 0.586	ABBOTT ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc IgM
	TEST DATE: 12/19/1996	TEST DATE: 12/3/1996	TEST DATE: 1/13/1997	TEST DATE: 12/16/1996	TEST DATE: 8/11/2010	TEST DATE: 12/17/1996	TEST DATE: 6/18/1998	TEST DATE: 8/11/2010	TEST DATE: 8/11/2010
6272-01	0.65	0.55	0.28	0.58	0.40	1.164	0.082	3.38	0.02
6272-02	0.63	0.68	0.74	1.39	1.03	1.264	0.082	3.18	0.01
6272-03	1.09	0.93	2.12	2.70	2.29	1.514	0.072	3.14	0.01
6272-04	1.49	1.43	4.63	3.86	3.01	1.469	0.082	3.49	0.01
6272-05	4.78	5.33	15.3	17.8	20.20	1.161	0.087	3.55	0.01
6272-06	14.9	13.5	36.8	43.6	56.40	1.195	0.100	3.32	0.01
6272-07	33.13	36.76	123.6	MAX.ABS.	197.00	1.306	0.077	3.39	0.01

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6274 Rev03
 Effective Date: 06/14/2021

ZeptoMetrix®

HBV Seroconversion Panel Donor No.61799 Part Number: HBV6274

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS Eci ANTI-HBs QUANT TEST DATE: 8/11/2010	ABBOTT HBeAg (EIA) OD Negative < 0.121 Positive ≥ 0.121 TEST DATE: 6/30/1998
6272-01	0.30	0.040
6272-02	1.30	0.038
6272-03	0.60	0.037
6272-04	0.10	0.039
6272-05	0.10	0.038
6272-06	0.40	0.046
6272-07	1.00	0.054

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6274 Rev03
Effective Date: 06/14/2021