

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No.60409 Part Number: HBV6276

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	BIOMERIEUX VIDAS HBsAg	BOEHR. MANNHEIM ELECSYS HBsAg	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ROCHE Cobas Core HBsAg II EIA	SORIN ETI-MAK 3 HBsAg
		TEST DATE: 11/17/1995	TEST DATE: UNK	TEST DATE: 1/16/1997	TEST DATE: 10/9/1997	TEST DATE: 12/18/1996	TEST DATE: 12/17/1996	TEST DATE: 1/16/1997
6276-01	10/12/95	4	NOT DETECTED	0.11	0.49	0.56	0.15	0.25
6276-02	10/15/95	3	NOT DETECTED	0.13	0.38	0.51	0.13	0.11
6276-03	10/19/95	3	<52	0.11	0.42	0.67	0.61	0.25
6276-04	10/22/95	6	NOT DETECTED	0.11	0.38	0.53	0.25	0.18
6276-05	10/26/95	6	<52	0.12	0.45	0.54	0.19	0.18
6276-06	10/29/95	5	92	0.17	0.45	0.55	0.21	0.22
6276-07	11/08/95	4	1182	0.66	1.28	0.61	11.47	0.74
6276-08	11/10/95	6	2253	1.13	1.90	0.68	1.57	1.18

PANEL MEMBER	ABBOTT MUREX HBsAg	ABBOTT PRISM HBsAg	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	ABBOTT AxSYM HBsAg	ABBOTT Austria II-125	ABBOTT Auszyme MC Dynamic	BEHRINGWERKE Enzygnost HBsAg Monoclonal II
	TEST DATE: 1/9/1997	TEST DATE: 8/27/2009	TEST DATE: 2/5/1997	TEST DATE: 8/27/2009	TEST DATE: 1/6/1997	TEST DATE: 12/3/1996	TEST DATE: 1/21/1997	TEST DATE: 12/19/1996
6276-01	0.61	0.80	0.46	0.50	0.54	0.31	0.21	0.19
6276-02	0.99	0.32	0.56	0.50	0.53	0.33	0.21	0.49
6276-03	0.52	0.73	0.50	0.53	0.54	0.35	0.55	0.18
6276-04	0.48	0.34	0.38	0.49	0.46	0.39	0.24	0.22
6276-05	0.58	0.77	0.41	0.49	0.51	0.36	0.33	0.33
6276-06	0.63	1.09	0.65	0.51	0.49	0.58	0.33	0.42
6276-07	0.72	2.16	1.99	1.23	0.90	0.02	1.42	0.67
6276-08	0.97	3.28	3.16	1.85	1.09	0.01	2.33	0.82

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6276 Rev03  
Effective Date: 06/23/2021

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No.60409 Part Number: HBV6276

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 12/16/1996	ORTHO AMERLITE HBsAg II TEST DATE: 1/13/1997	ORTHO VITROS ECi HBsAg TEST DATE: UNK	ABBOTT IMx HBsAg Reagent Pack TEST DATE: 3/13/1997	ORTHO VITROS ECi HBeAg TEST DATE: UNK	ABBOTT HBeAg (EIA) OD TEST DATE: 12/19/1996	DIASORIN HBc IgM TEST DATE: 10/5/2011
6276-01	0.14	0.09	0.13	0.51	0.22	0.045	0.17
6276-02	0.38	0.07	0.12	0.57	0.22	Not Tested	0.17
6276-03	0.20	0.08	0.14	0.45	0.19	0.047	0.17
6276-04	0.26	0.06	0.13	0.47	0.20	0.057	0.17
6276-05	0.55	0.08	0.13	0.43	0.21	0.043	0.17
6276-06	0.32	0.09	0.16	0.47	0.22	0.048	0.17
6276-07	1.01	0.23	0.64	0.64	0.21	0.045	0.17
6276-08	1.42	0.47	1.11	0.84	0.21	0.046	0.17

PANEL MEMBER	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL (EIA) TEST DATE: 12/19/1996	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBe TEST DATE: UNK	ORTHO VITROS ECi HBeAg TEST DATE: UNK
6276-01	0.04	3.49	0.404	1.4	0.15	0.22
6276-02	0.03	3.02	0.455	1.8	0.16	0.22
6276-03	0.03	3.38	0.410	0.3	0.17	0.19
6276-04	0.03	3.02	0.463	4.4	0.17	0.2
6276-05	0.03	3.04	0.460	1.2	0.16	0.21
6276-06	0.03	3.23	0.555	1.5	0.17	0.22
6276-07	0.03	3.18	0.453	2.7	0.16	0.21
6276-08	0.04	3.02	0.426	3.1	0.16	0.21

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6276 Rev03  
Effective Date: 06/23/2021