

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No.63291 Part Number: HBV6277

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 3/05/1997	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	RESEARCH USE PCR  TEST DATE: 10/29/1997	BIOMERIEUX VIDAS HBsAg (>0.13 reactive)  TEST DATE: 10/21/1997	BOEHRINGER MANNHEIM ELECSYS HBsAg  TEST DATE: 10/8/1997	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II  TEST DATE: 10/7/1997	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: 10/30/1997	ROCHE Cobas Core HBsAg II EIA  TEST DATE: 10/9/1997
6277-01	01/12/97	0	NOT DETECTED	<100	0.01	0.46	0.34	0.08	0.47
6277-02	01/16/97	4	NOT DETECTED	<100	0.02	0.42	0.44	0.19	0.41
6277-03	02/02/97	3	<52	<100	0.02	0.48	0.42	0.22	0.35
6277-04	02/07/97	8	271	100	0.03	0.58	0.40	0.16	0.69
6277-05	02/09/97	10	175	100	0.03	0.65	0.35	0.16	0.38
6277-06	02/14/97	8	635	200	0.07	1.08	0.52	0.60	0.69
6277-07	02/16/97	8	983	200	0.08	1.31	0.60	0.98	0.69
6277-08	02/21/97	2	2587	500	0.24	3.00	0.89	2.98	1.40
6277-09	02/23/97	6	3739	2000	0.36	4.30	1.04	4.43	2.07
6277-10	02/28/97	4	14885	3000	1.35	15.64	3.32	17.7	7.20
6277-11	03/02/97	9	15950	3000	2.16	25.75	4.70	26.91	10.51

PANEL MEMBER	SORIN ETI-MAK 3 HBsAg  TEST DATE: 11/12/1997	MUREX HBsAg  TEST DATE: 10/29/1997	ABBOTT PRISM HBsAg  TEST DATE: UNK	ABBOTT PRISM HBsAg  TEST DATE: 11/11/1997	ABBOTT MUREX HBsAg Version 3  TEST DATE: 8/27/2009	ABBOTT AxSYM HBsAg Reagent Pack  TEST DATE: 10/22/1997	ABBOTT IMx HBsAg Reagent Pack  TEST DATE: 10/30/1997	BIOTEST ELISA HBsAg  TEST DATE: 11/5/1997	ORTHO VITROS ECi HBsAg  TEST DATE: UNK
6277-01	0.28	0.37	0.71	0.19	0.49	0.43	0.45	0.14	0.11
6277-02	0.24	0.44	0.57	0.27	0.46	0.46	0.42	0.11	0.12
6277-03	0.43	0.37	0.41	0.31	0.50	0.51	0.38	0.12	0.12
6277-04	0.41	0.45	0.69	0.43	0.56	0.54	0.49	0.22	0.16
6277-05	0.31	0.64	0.84	0.47	0.61	0.55	0.45	0.22	0.18
6277-06	0.52	0.50	1.96	1.25	1.04	0.68	0.56	0.28	0.44
6277-07	0.60	0.73	2.53	1.63	1.36	0.98	0.70	0.42	0.71
6277-08	1.44	1.12	6.82	4.71	3.29	1.71	1.32	1.30	2.87
6277-09	1.83	1.53	10.48	6.56	5.32	2.49	1.77	1.83	4.12
6277-10	7.87	5.16	34.99	24.27	19.17	8.26	6.54	6.70	20.1
6277-11	9.41	7.59	54.23	36.66	26.25	13.32	8.97	10.00	34.1

PANEL MEMBER	BEHRINGWERKE Enzygnost HBsAg Monoclonal II  TEST DATE: 10/16/1997	ABBOTT ANTI-HBc TOTAL  TEST DATE: 12/5/1997	ORTHO VITROS ECi ANTI-HBc TOTAL  TEST DATE: UNK	DIASORIN HBc IgM  TEST DATE: 10/5/2011	ORTHO VITROS ECi ANTI-HBc IgM  TEST DATE: UNK	ORTHO VITROS ECi HBeAg  TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBe  TEST DATE: UNK	ABBOTT ANTI-HBs QUANT  TEST DATE: 12/5/1997	ORTHO VITROS ECi ANTI-HBs QUANT  TEST DATE: UNK
6277-01	0.24	0.044	3.19	0.17	0.01	0.22	0.13	0.079	.4
6277-02	0.29	0.031	3.04	0.13	0.01	0.22	0.14	0.127	.4
6277-03	0.28	0.040	3.22	0.17	0.01	0.24	0.14	0.111	.4
6277-04	0.39	0.038	3.22	0.17	0.01	0.23	0.14	0.111	.3
6277-05	0.35	0.025	3.33	0.17	0.01	0.21	0.15	0.746	.8
6277-06	0.39	0.100	3.47	0.17	0.01	0.23	0.13	0.111	1
6277-07	0.47	0.023	3.37	0.17	0.01	0.21	0.14	0.143	0
6277-08	0.93	0.038	3.47	0.17	0.01	0.23	0.14	0.127	.3
6277-09	1.16	0.027	3.35	0.17	0.01	0.23	0.14	0.095	.6
6277-10	4.03	0.034	3.46	0.17	0.01	0.29	0.14	0.095	.4
6277-11	6.63	0.021	3.38	0.17	0.01	0.29	0.14	0.079	.4

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6277 Rev03  
Effective Date: 06/23/2021