



**HBV Seroconversion Panel Donor No. 62825**  
**Part Number: HBV6283**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 1/27/1998	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	RESEARCH USE PCR (copies/ml)  TEST DATE: 6/2/1998	BIOMERIEUX VIDAS HBsAg  TEST DATE: 4/22/1998	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems  TEST DATE: 4/27/1998	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II  TEST DATE: 4/23/1998	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: 3/18/1998
6283-01	11/05/96	5	NOT DETECTED	100	0.16	0.13	0.38	0.15
6283-02	11/07/96	0	NOT DETECTED	<100	0.13	0.36	0.38	0.15
6283-03	11/13/96	3	<52	<100	0.17	0.33	0.39	0.10
6283-04	11/16/96	5	NOT DETECTED	100	0.16	0.38	0.39	0.07
6283-05	11/21/96	8	70	100	0.13	0.38	0.35	0.21
6283-06	11/24/96	6	82	500	0.18	0.29	0.32	0.15
6283-07	12/01/96	5	1182	2000	0.32	0.58	0.64	0.60
6283-08	12/04/96	10	2836	6000	0.68	0.80	1.15	1.30
6283-09	12/12/96	8	22017	100000	7.57	6.32	9.15	14.14
6283-10	12/15/96	12	71245	160000	18.19	13.71	17.57	28.99
6283-11	12/20/96	15	449526	600000	56.76	54.65	MAX.ABS.	80.91

PANEL MEMBER	ROCHE Cobas Core HBsAg II EIA  TEST DATE: 4/23/1998	GENETIC SYSTEMS HBsAg (2.0)  TEST DATE: 2/4/1998	ABBOTT PRISM HBsAg  TEST DATE: 8/27/2009	ABBOTT MUREX HBsAg Version 3  TEST DATE: 8/27/2009	BEHRINGWERKE Enzygnost HBsAg Monoclonal II  TEST DATE: 5/4/1998	BIOTEST ELISA HBsAg  TEST DATE: 4/6/1998	ABBOTT AxSYM HBsAg  TEST DATE: 3/19/1998	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 3/31/1998
6283-01	0.24	0.170	0.69	0.65	0.51	0.47	0.54	0.07
6283-02	0.20	0.159	0.71	0.70	0.51	0.32	0.52	0.07
6283-03	0.17	0.205	0.57	0.67	0.53	0.32	0.57	0.03
6283-04	0.15	0.227	0.26	0.53	0.46	0.28	0.53	0.07
6283-05	0.22	0.148	0.82	0.58	0.40	0.27	0.48	0.03
6283-06	0.15	0.170	0.88	0.60	0.35	0.25	0.49	0.10
6283-07	1.02/0.55	0.284	1.62	0.99	0.46	0.39	0.70	0.53
6283-08	1.15/0.84	0.420	4.34	2.02	0.60	0.64	0.97	1.25
6283-09	11.81	4.670	34.66	14.97	2.78	5.11	5.37	13.62
6283-10	23.22	10.216	62.33	27.79	5.35	9.53	10.70	29.74
6283-11	MAX.ABS.	24.545	180.84	32.67	21.28	24.09	35.35	over

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6283 Rev04  
 Effective Date: 06/23/2021



**HBV Seroconversion Panel Donor No. 62825**  
**Part Number: HBV6283**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: 4/30/1998	ABBOTT AxSYM Core-M TEST DATE: UNK	ABBOTT Core-M ANTI-HBc IgM TEST DATE: UNK	ABBOTT HBeAg TEST DATE: UNK	ABBOTT Core-ANTI-HBc TOTAL TEST DATE: UNK	ABBOTT AxSYM Core (Co/S) TEST DATE: UNK	SORIN ETI-MAK 3 TEST DATE: 5/26/1998	SORIN (INCSTAR) HBsAg TEST DATE: UNK
6283-01	0.28	0.04	0.057	NOT TESTED	0.43	0.49	0.38	0.235
6283-02	0.24	0.05	0.057	0.269	0.30	0.53	0.43	0.157
6283-03	0.41	0.04	0.057	0.192	0.31	0.51	.055	0.196
6283-04	0.14	0.04	0.057	0.223	0.40	0.51	0.57	0.196
6283-05	0.10	0.04	0.057	0.246	0.34	0.51	0.41	0.157
6283-06	0.13	0.04	0.057	0.215	0.51	0.51	0.49	0.118
6283-07	0.31	0.04	0.075	0.223	0.42	0.51	0.37	0.157
6283-08	0.47	0.05	0.057	0.223	0.41	0.51	1.00	0.275
6283-09	6.40	0.04	0.057	0.254	0.53	0.51	17.86	2.333
6283-10	7.82	0.04	0.113	0.315	0.40	0.51	23.60	4.235
6283-11	35.35	0.04	0.075	0.977	0.47	0.53	59.67	9.059

PANEL MEMBER	SORIN ETI-Core IGM2 TEST DATE: UNK	SORIN ETI-AB CoreK2 (Co/S) TEST DATE: UNK	ORTHO VITROS ECi HBsAg TEST DATE: 10/20/2015	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 10/20/2015	ORTHO VITROS ECi HBeAg TEST DATE: 10/20/2015	ORTHO VITROS ECi ANTI-HBe TEST DATE: 10/20/2015	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 10/20/2015	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 10/20/2015
6283-01	0.00	0.16	0.05	3.31	0.34	0.15	0.01	1.6
6283-02	0.00	0.18	0.04	3.19	0.34	0.15	0.01	1.4
6283-03	0.00	0.19	0.05	3.08	0.3	0.15	0.02	2.4
6283-04	0.00	0.18	0.05	3.29	0.31	0.16	0.01	1
6283-05	0.00	0.18	0.04	3.23	0.3	0.16	0.01	2.2
6283-06	0.00	0.18	0.05	3.23	0.26	0.14	0.01	1.8
6283-07	0.00	0.17	0.14	3.09	0.2	0.15	0.01	1.5
6283-08	0.06	0.16	0.3	3.31	0.18	0.16	0.01	0.1
6283-09	0.00	0.16	12.8	3.34	0.16	0.14	0.01	1.6
6283-10	0.00	0.19	33.8	3.28	0.23	0.14	0.01	0.9
6283-11	0.00	0.18	162	3.25	3.46	0.12	0.01	0.8

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6283 Rev04  
 Effective Date: 06/23/2021