

ZeptoMetrix®

HBV Seroconversion Panel Donor No.63659 Part Number: HBV6289

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 4/28/1997	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	RESEARCH USE PCR (copies/ml) TEST DATE: 6/14/1998	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: 2/6/1998	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 4/6/1998	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: 5/5/1998	ABBOTT AxSYM HBsAg TEST DATE: 3/19/1998
6289-01	03/10/97	8	NOT DETECTED	<100	Negative	0.130	0.23	0.18	0.66
6289-02	03/12/97	8	NOT DETECTED	<100	Negative	0.139	0.10	0.18	0.44
6289-03	03/19/97	7	NOT DETECTED	<100	Negative	0.139	0.10	0.18	0.51
6289-04	03/21/97	9	NOT DETECTED	<100	Negative	0.148	0.07	0.18	0.47
6289-05	03/26/97	5	NOT DETECTED	100	Negative	0.120	0.16	0.34	0.44
6289-06	03/28/97	1	<52	100	Not Tested	0.130	0.10	0.28	0.43
6289-07	04/10/97	4	1238	7300	Not Tested	0.565	1.50	0.52	0.94
6289-08	04/15/97	6	7812	29000	Not Tested	1.657	2.80	1.26	1.70
6289-09	04/24/97	7	51612	160000	Not Tested	11.009	47.41	15.06	13.81
6289-10	04/28/97	6	18237	460000	Not Tested	16.009	MAX ABS.	33.24	27.37

PANEL MEMBER	ABBOTT PRISM HBsAg TEST DATE: 8/27/2009	BIOTEST ELISA HBsAg TEST DATE: 4/6/1998	BEHRINGWERKE Enzygnost HBsAg Monoclonal II TEST DATE: 5/4/1998	Abbott MUREX HBsAg Version 3 TEST DATE: 8/27/2009	BIOMERIEUX VIDAS HBsAg TEST DATE: 4/30/1998	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems TEST DATE: 4/27/1998	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 4/23/1998	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 3/18/1998
6289-01	0.77	3.49	0.38	0.51	0.32	0.13	0.46	0.29
6289-02	0.31	0.26	0.24	0.51	0.12	0.00	0.34	0.10
6289-03	0.63	0.32	0.33	0.49	0.26	0.03	0.52	0.38
6289-04	0.47	0.29	0.34	0.47	0.25	0.15	0.51	0.18
6289-05	0.60	0.22	0.24	0.47	0.17	0.00	0.37	0.01
6289-06	0.66	0.22	0.23	0.50	0.15	0.10	0.38	0.01
6289-07	2.17	0.63	1.05	1.33	0.94	0.42	0.92	1.22
6289-08	6.63	0.98	1.28	3.94	1.94	1.08	1.80	2.78
6289-09	68.67	10.83	15.09	30.51	23.05	17.63	13.83	37.58
6289-10	111.97	17.06	30.31	31.33	44.18	41.53	27.34	65.94

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6289 Rev02
Effective Date: 06/14/2021



HBV Seroconversion Panel Donor No.63659
Part Number: HBV6289

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ROCHE Cobas Core HBsAg II EIA TEST DATE: 4/27/1998	SORIN (INCSTAR) HBsAg TEST DATE: UNK	SORIN ETI-Core IGMK2 TEST DATE: 5/11/1998	SORIN ETI-AB CoreK2 TEST DATE: 5/7/1998	ABBOTT AxSYM Core-M TEST DATE: 4/7/1998	ABBOTT AxSYM Core TEST DATE: 4/7/1998	ORTHO VITROS Eci HBsAg TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK
6289-01	0.20	0.116	0.25	0.19	0.09	0.56	0.06	0.45
6289-02	0.65	0.173	0.22	0.18	0.08	0.56	0.06	0.52
6289-03	0.41	0.173	0.27	0.18	0.08	0.51	0.06	0.45
6289-04	0.59	0.173	0.25	0.18	0.08	0.56	0.05	0.45
6289-05	0.13	0.116	0.25	0.18	0.08	0.54	0.06	0.45
6289-06	1.09/0.11	0.173	0.26	0.18	0.07	0.52	0.08	0.39
6289-07	2.46/1.04	0.462	0.24	0.18	0.08	0.56	0.49	0.42
6289-08	3.09/2.12	0.757	0.22	0.17	0.08	0.52	2.85	0.39
6289-09	41.19	6.590	0.23	0.18	0.08	0.52	50.8	0.39
6289-10	60.76	12.139	0.26	0.19	0.09	0.53	Not Tested	0.39

PANEL MEMBER	ORTHO VITROS Eci ANTI-HBc TOTAL TEST DATE: UNK	ABBOTT ANTI-HBc IgM TEST DATE: UNK	ORTHO VITROS Eci ANTI-HBc IgM TEST DATE: UNK	ABBOTT HBeAg TEST DATE: UNK	ORTHO VITROS Eci HBeAg TEST DATE: UNK	ORTHO VITROS Eci ANTI-HBe TEST DATE: UNK	ORTHO VITROS Eci ANTI-HBs QUANT TEST DATE: UNK	ORTHO VITROS Eci ANTI-HBs QUANT TEST DATE: UNK
6289-01	3.15	0.127	Not Tested	0.292	0.29	Not Tested	Not Tested	0.06
6289-02	2.88	0.109	0.04	0.247	0.28	0.15	0.10	0.06
6289-03	3.00	0.127	0.04	0.315	0.27	0.17	0.20	0.06
6289-04	2.83	0.127	0.05	0.231	0.26	0.16	0.60	0.05
6289-05	3.17	0.127	0.04	0.270	0.27	0.15	1.50	0.06
6289-06	2.91	0.109	Not Tested	0.292	Not Tested	Not Tested	Not Tested	0.08
6289-07	3.00	0.127	0.06	0.348	0.27	0.16	0.40	0.49
6289-08	3.00	0.127	0.04	0.427	0.32	Not Tested	1.40	2.85
6289-09	2.64	0.109	0.05	0.551	0.7	Not Tested	0.50	50.8
6289-10	Not Tested	0.145	Not Tested	0.753	Not Tested	Not Tested	Not Tested	Not Tested

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6289 Rev02
 Effective Date: 06/14/2021