

ZeptoMetrix®

HBV Seroconversion Panel Donor No.64121 Part Number: HBV6291

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L) TEST DATE: 2/4/1998	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	RESEARCH USE PCR (copies/ml) TEST DATE: 6/8/1998	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: 2/6/1998	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 4/6/1998	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: 5/15/1998	ABBOTT AxSYM HBsAg TEST DATE: 3/19/1998
6291-01	05/14/97	5	NOT DETECTED	100	0.139	0.65	0.13	0.48
6291-02	05/16/97	4	<52	100	0.167	0.10	0.13	0.46
6291-03	05/22/97	5	78	100	0.157	0.33	0.20	0.49
6291-04	05/24/97	2	59	500	0.176	0.20	0.23	0.42
6291-05	05/29/97	6	277	500	0.185	0.55	0.26	0.59
6291-06	06/10/97	8	2709	3400	1.083	2.87	0.76	1.41
6291-07	06/17/97	3	19176	63000	5.028	11.32	2.97	4.40
6291-08	06/19/97	8	14546	63000	6.065	14.64	4.29	5.99

PANEL MEMBER	ABBOTT PRISM HBsAg TEST DATE: 8/27/2009	BIOTEST ELISA HBsAg TEST DATE: 4/6/1998	BEHRINGWERKE Enzygnost HBsAg Monoclonal II TEST DATE: 5/4/1998	ABBOTT MUREX HBsAg Version 3 TEST DATE: 8/27/2009	BIOMERIEUX VIDAS HBsAg TEST DATE: 4/30/1998	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems TEST DATE: 4/27/1998	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 4/23/1998	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 3/18/1998	ROCHE Cobas Core HBsAg II EIA TEST DATE: 4/27/1998
6291-01	1.20	0.21	0.34	0.50	0.07	0.05	0.52	0.07	0.20
6291-02	1.23	0.33	0.55	0.60	0.11	0.00	0.38	0.01	0.78
6291-03	1.05	0.28	0.35	0.59	0.13	0.00	0.44	0.10	0.15
6291-04	1.20	0.32	0.38	0.57	0.11	0.08	0.42	0.07	0.52
6291-05	1.89	0.35	0.40	0.63	0.26	0.10	0.44	0.22	0.56
6291-06	6.29	1.05	1.01	2.80	1.50	0.95	1.71	2.58	2.74
6291-07	23.26	4.32	3.49	11.76	5.44	4.32	5.51	11.37	10.91
6291-08	17.47	5.49	4.64	9.82	8.14	6.28	6.52	15.14	14.19

PANEL MEMBER	SORIN (INCSTAR) HBsAg TEST DATE: UNK	ORTHO VITROS ECi HBsAg TEST DATE: 6/02/2016	SORIN ETI-Core IGMK2 TEST DATE: 5/11/1998	SORIN ETI-AB CoreK2 (Co/S) TEST DATE: 5/7/1998	ABBOTT AxSYM Core-M TEST DATE: 4/7/1998	ABBOTT AxSYM Core (Co/S) TEST DATE: 4/7/1998	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 6/02/2016	ORTHO VITROS ECi HBeAg TEST DATE: 6/02/2016	ABBOTT HBeAg TEST DATE: UNK
6291-01	0.173	0.05	0.49	0.21	0.06	0.50	2.82	0.26	0.303
6291-02	0.173	0.05	0.54	0.21	0.06	0.48	2.90	0.22	0.303
6291-03	0.290	0.05	0.46	0.21	0.06	0.49	2.77	0.24	0.326
6291-04	0.231	0.05	0.49	0.20	0.06	0.46	3.01	0.25	0.281
6291-05	0.347	0.09	0.44	0.22	0.06	0.49	2.48	0.23	0.270
6291-06	0.578	2.04	0.33	0.22	0.06	0.48	2.69	0.24	0.348
6291-07	1.965	17.9	0.36	0.21	0.07	0.49	2.76	0.31	0.303
6291-08	2.197	20.1	0.45	0.21	0.06	0.47	2.58	0.36	0.438

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6291 Rev02
Effective Date: 06/24/2021



HBV Seroconversion Panel Donor No.64121
Part Number: HBV6291

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi ANTI-HBe TEST DATE: 6/02/2016	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ABBOTT ANTI-HBc IgM TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 6/02/2016	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 6/02/2016
6291-01	0.15	0.45	0.145	0.04	5.30
6291-02	0.15	0.48	0.164	0.04	2.00
6291-03	0.15	0.45	0.145	0.06	3.70
6291-04	0.15	0.39	0.145	0.05	2.20
6291-05	0.19	0.45	0.164	0.07	2.90
6291-06	0.16	0.45	0.200	0.07	2.80
6291-07	0.15	0.42	0.145	0.05	2.10
6291-08	0.16	0.37	0.145	0.05	2.60

www.h-h-c.com

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com