



HBV Seroconversion Panel Donor No.64006
Part Number: HBV6292

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	RESEARCH USE PCR (copies/ml)	ROCHE CAP-G/CTM HBV (cp/mL)	GENETIC SYSTEMS HBsAg (2.0)	ABBOTT Auszyme MC Dynamic HBsAg
		TEST DATE: 2/4/1998	TEST DATE: UNK	TEST DATE: 6/13/1998	TEST DATE: 6/2/2005	TEST DATE: 2/6/1998	TEST DATE: 4/6/1998
6292-01	04/14/97	5	NOT DETECTED	<100	NEGATIVE	0.130	0.46
6292-02	04/16/97	4	NOT DETECTED	<100	NEGATIVE	0.093	0.07
6292-03	04/23/97	5	<52	200	NEGATIVE	0.269	0.13
6292-04	04/25/97	0	NOT DETECTED	100	NEGATIVE	0.250	0.10
6292-05	05/05/97	8	311	550	641	0.287	0.00
6292-06	05/07/97	3	231	1200	1090	0.241	0.10
6292-07	05/13/97	6	1709	2600	7550	0.463	1.60
6292-08	05/19/97	2	7994	21000	54300	0.778	1.83
6292-09	05/26/97	7	22530	66000	111000	7.731	20.58
6292-10	05/28/97	3	59259	150000	166000	11.769	42.03
6292-11	06/02/97	5	296999	750000	959000	23.657	MAX ABS.
6292-12	06/05/97	4	1029804	1900000	2350000	20.222	MAX ABS.

PANEL MEMBER	ABBOTT Auszyme Monoclonal HBsAg	ABBOTT AxSYM HBsAg	ABBOTT PRISM HBsAg	BIOTEST ELISA HBsAg	BEHRINGWERKE Enzygnost HBsAg Monoclonal II	ABBOTT MUREX HBsAg Version 3	BIOMERIEUX VIDAS HBsAg	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems
	TEST DATE: 5/5/1998	TEST DATE: 3/19/1998	TEST DATE: 8/27/2009	TEST DATE: 4/6/1998	TEST DATE: 5/4/1998	TEST DATE: 8/27/2009	TEST DATE: 4/30/1998	TEST DATE: 4/27/1998
6292-01	0.43	0.62	0.70	0.25	0.51	0.53	0.30	0.20
6292-02	0.21	0.46	0.61	0.27	0.34	0.57	0.07	0.00
6292-03	0.15	0.46	0.82	0.28	0.39	0.62	0.11	0.00
6292-04	0.15	0.41	0.66	0.31	0.35	0.56	0.07	0.08
6292-05	0.18	0.48	0.67	0.31	0.31	0.69	0.17	0.00
6292-06	0.18	0.51	0.96	0.35	0.35	0.72	0.21	0.00
6292-07	0.55	0.71	2.22	0.51	0.56	1.37	0.51	0.32
6292-08	1.22	1.22	4.21	0.90	1.06	3.01	1.62	1.20
6292-09	6.73	8.83	32.77	7.87	8.48	20.61	12.48	10.45
6292-10	16.07	13.33	54.70	13.54	12.43	29.88	27.54	18.17
6292-11	MAX ABS.	36.36	149.33	33.78	35.58	31.64	73.21	85.68
6292-12	MAX ABS.	47.22	174.61	MAX ABS.	36.76	30.78	99.81	158.82

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6292 Rev03
 Effective Date: 06/24/2021



HBV Seroconversion Panel Donor No.64006
Part Number: HBV6292

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 4/23/1998	ORTHO HBsAg ELISA Test- system 3 TEST DATE: 3/18/1998	ORTHO VITROS ECi HBsAg TEST DATE: 6/02/2016	ROCHE Cobas Core HBsAg II EIA TEST DATE: 4/27/1998	SORIN (INCSTAR) HBsAg TEST DATE: UNK	SORIN ETI-Core IGMK2 TEST DATE: UNK	SORIN ETI-AB CoreK2 (Co/S) TEST DATE: UNK	ABBOTT AxSYM Core-M TEST DATE: UNK	ABBOTT AxSYM Core (Co/S) TEST DATE: UNK
6292-01	0.62	0.69	0.06	0.55	0.173	0.51	0.19	0.05	0.48
6292-02	0.35	0.13	0.05	0.20	0.173	0.44	0.20	0.04	0.49
6292-03	0.35	0.13	0.07	0.22	0.173	0.40	0.20	0.05	0.50
6292-04	0.38	0.13	Not Tested	0.25	0.173	0.48	0.19	0.04	0.50
6292-05	0.84	0.19	0.09	0.27	0.173	0.55	0.20	0.04	0.52
6292-06	0.51	0.28	0.10	0.35	0.173	0.59	0.20	0.05	0.51
6292-07	0.86	1.07	0.47	0.95	0.289	0.49	0.21	0.05	0.51
6292-08	1.51	2.55	2.03	2.62	0.520	0.54	0.22	0.04	0.49
6292-09	12.01	23.64	27.20	22.93	3.931	0.49	0.21	0.05	0.51
6292-10	17.75	38.96	44.30	38.83	5.896	0.46	0.21	0.05	0.53
6292-11	MAX ABS.	MAX ABS	168	MAX ABS.	12.717	0.52	0.22	0.05	0.52
6292-12	MAX ABS.	MAX ABS.	295	MAX ABS.	11.850	0.48	0.20	0.04	0.57

PANEL MEMBER	ABBOTT ANTI-HBc IgM TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 6/02/2016	ABBOTT HBeAg TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 6/02/2016	ORTHO VITROS ECi HBeAg TEST DATE: 6/02/2016	ORTHO VITROS ECi ANTI-HBe TEST DATE: 6/02/2016	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 6/02/2016
6292-01	0.127	0.03	0.337	0.45	3.38	Not Tested	Not Tested	Not Tested
6292-02	0.109	0.06	0.326	0.42	3.44	0.30	0.14	10.7
6292-03	0.109	Not Tested	0.270	0.34	3.48	QNS	0.14	Not Tested
6292-04	0.109	Not Tested	0.315	0.52	3.31	QNS	QNS	Not Tested
6292-05	0.127	0.05	0.281	0.45	3.85	0.26	0.14	9.50
6292-06	0.127	0.04	0.236	0.52	3.36	0.28	0.13	7.40
6292-07	0.109	Not Tested	0.910	0.42	3.29	0.34	0.14	Not Tested
6292-08	0.109	0.04	0.292	0.48	3.23	0.31	Not Tested	7.60
6292-09	0.182	0.10	0.663	0.42	3.23	1.41	Not Tested	7.80
6292-10	0.164	0.11	1.056	0.56	3.25	3.19	Not Tested	7.20
6292-11	0.145	0.08	3.123	0.56	Not Tested	20.5	0.10	5.9
6292-12	0.127	0.08	6.348	0.52	3.17	66.8	0.06	9.30

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6292 Rev03
 Effective Date: 06/24/2021