



HBV Seroconversion Panel Donor No.64132
Part Number: HBV6293

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT	RESEARCH	GENETIC	ABBOTT	ABBOTT	BEHRINGWERKE
		Normal Range (10-45u/L)	HBV REAL TIME CP/ML	USE PCR (copies/ml)	SYSTEMS HBsAg (2.0)	Auszyme MC Dynamic HBsAg	Auszyme Monoclonal HBsAg	Enzygnost HBsAg Monoclonal II
		TEST DATE: 7/21/1997	TEST DATE: UNK	TEST DATE: 6/2/1998	TEST DATE: 2/6/1998	TEST DATE: 4/6/1998	TEST DATE: 5/5/1998	TEST DATE: 5/4/1998
6293-01	05/20/1997	8	<52	500	0.130	0.65	0.37	0.06
6293-02	05/26/1997	10	341	500	0.204	0.16	0.31	0.28
6293-03	05/29/1997	10	333	750	0.213	0.03	0.15	0.29
6293-04	06/04/1997	14	1389	5100	0.519	0.55	0.46	0.46
6293-05	06/12/1997	10	32565	42000	3.880	6.36	4.56	2.81
6293-06	06/18/1997	14	247033	180000	12.824	48.65	16.99	17.56
6293-07	06/24/1997	9	1831278	2200000	21.981	MAX ABS.	MAX ABS.	37.26

PANEL MEMBER	ABBOTT	ABBOTT	BIOMERIEUX	SORIN	ORTHO	BIOTEST	BOEHR.	ORGANON
	AxSYM HBsAg	PRISM HBsAg	VIDAS HBsAg	(INCSTAR) HBsAg	VITROS ECi HBsAg	ELISA HBsAg	MANNHEIM HBsAg Enzymun-test ES Systems	TEKNIKA Hepanostika HBsAg Uni-Form II
	TEST DATE: 3/19/1998	TEST DATE: UNK	TEST DATE: 4/30/1998	TEST DATE: UNK	TEST DATE: 6/02/2016	TEST DATE: 4/6/1998	TEST DATE: 4/27/1998	TEST DATE: 4/23/1998
6293-01	0.66	0.84	0.32	0.173	0.08	0.47	0.22	0.56
6293-02	0.47	0.77	0.13	0.173	0.08	0.31	0.05	0.49
6293-03	0.57	1.02	0.14	0.231	0.08	0.35	0.00	0.51
6293-04	0.76	2.28	0.51	0.347	0.39	0.52	0.35	0.79
6293-05	3.14	2.08	5.36	1.728	11.5	3.59	3.17	4.58
6293-06	15.12	104.57	25.48	6.532	66.9	18.15	21.15	23.73
6293-07	45.66	268.91	103.8	11.329	401	34.50	123.50	MAX ABS.

PANEL MEMBER	ORTHO	ABBOTT	ROCHE	ABBOTT	ORTHO	SORIN	SORIN	ABBOTT	ABBOTT
	HBsAg ELISA Testsystem 3	MUREX HBsAg Version 3	Cobas Core HBsAg II EIA	ANTI-HBc IgM	VITROS ECi ANTI-HBc IgM	ETI-Core IGMK2	ETI-AB CoreK2 (Co/S)	AxSYM Core-M	AxSYM Core-M (Co/S)
	TEST DATE: 3/18/1998	TEST DATE: UNK	TEST DATE: 4/27/1998	TEST DATE: UNK	TEST DATE: 6/02/2016	TEST DATE: 5/11/1998	TEST DATE: 5/7/1998	TEST DATE: 4/7/1998	TEST DATE: 4/7/1998
6293-01	0.49	0.51	0.35	0.164	0.06	0.45	0.30	0.08	0.52
6293-02	0.01	0.54	0.80	0.164	0.07	0.51	0.26	0.07	0.49
6293-03	0.07	0.57	0.91	0.164	Not Tested	0.44	0.25	0.07	0.48
6293-04	0.66	1.22	2.00	0.145	0.06	0.40	0.28	0.07	0.51
6293-05	7.40	9.27	38.87	0.164	0.06	0.48	0.27	0.06	0.51
6293-06	42.53	30.88	OVER	0.164	0.07	0.55	0.29	0.07	0.49
6293-07	MAX ABS.	31.10	MAX ABS.	0.164	0.06	0.59	0.32	0.07	0.51

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6293 Rev03
 Effective Date: 06/24/2021



HBV Seroconversion Panel Donor No.64132
Part Number: HBV6293

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi HBeAg TEST DATE: 6/02/2016	ABBOTT HBeAg TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 6/02/2016	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 6/02/2016	ORTHO VITROS ECi ANTI-HBe TEST DATE: 6/02/2016
6293-01	0.29	0.348	2.43	0.56	0.0	0.17
6293-02	0.28	0.247	2.16	0.61	0.0	0.20
6293-03	0.28	0.337	2.46	0.52	Not Tested	Not Tested
6293-04	0.27	0.270	2.62	0.56	0.10	0.15
6293-05	0.37	0.315	2.40	Not Tested	0.0	0.17
6293-06	1.74	0.708	2.43	Not Tested	0.2	0.16
6293-07	21.2	2.787	2.49	Not Tested	0.0	0.11

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6293 Rev03
Effective Date: 06/24/2021