

ZeptoMetrix®

HBV Seroconversion Panel Donor No.66481 Part Number: HBV9072

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT HBV REAL TIME CP/ML	ABBOTT IMx HBsAg (V2) Reagent Pack	ORTHO VITROS HBsAg Reagent Pack	ABBOTT AxSYM HBsAg (V2) Reagent Pack	ABBOTT PRISM HBsAg	ABBOTT Auszyme MC Dynamic HBsAg
		TEST DATE: 8/1/1998	TEST DATE: UNK	TEST DATE: 6/23/1999	TEST DATE: UNK	TEST DATE: 6/23/1999	TEST DATE: UNK	TEST DATE: 7/26/1999
9072-01	09/30/97	7	NOT DETECTED	0.550	0.120	0.430	0.63	0.030
9072-02	10/04/97	11	86	0.470	0.130	0.430	0.61	0.000
9072-03	10/07/97	12	NOT DETECTED	0.450	0.140	0.400	0.48	0.070
9072-04	10/16/97	9	NOT DETECTED	0.660	0.150	0.410	0.54	0.030
9072-05	10/19/97	7	1358	0.450	0.150	0.420	0.64	0.030
9072-06	10/23/97	10	439	0.460	0.130	0.400	0.40	0.070
9072-07	10/26/97	5	NOT DETECTED	0.490	0.130	0.460	0.34	0.030
9072-08	12/15/97	12	579	0.500	0.130	0.440	0.73	0.070
9072-09	01/09/98	3	NOT DETECTED	0.470	0.120	0.390	0.71	0.140
9072-10	01/14/98	8	NOT DETECTED	0.520	0.140	0.440	0.65	0.170
9072-11	01/16/98	4	88	0.540	0.110	0.430	1.33	0.310
9072-12	02/05/98	9	23591	1.070	0.140	0.860	6.05	0.870
9072-13	02/12/98	12	409972	19.850	0.150	12.760	105.17	36.420
9072-14	02/20/98	13	7460267	91.570	0.470	74.530	407.32	MAX ABS
9072-15	02/24/98	10	83705574	115.64	3.450	110.560	506.99	MAX ABS
9072-16	03/01/98	11	579101085	124.100	22.200	122.770	743.13	MAX ABS
9072-17	03/03/98	8	1789593439	145.260	26.100	119.260	831.69	MAX ABS
9072-18	03/08/98	9	>3410000000	135.860	32.700	109.540	1066.53	MAX ABS

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV9072 Rev02
Effective Date: 09/22/2021



**HBV Seroconversion Panel Donor No.66481
Part Number: HBV9072**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT Auszyme MC HBsAg TEST DATE: 5/27/1997	BIOTEST ELISA HBsAg TEST DATE: 6/22/1999	DADE BEHRING Enzygnost HbsAg 5.0 TEST DATE: 6/7/1999	BIOMERIEUX VIDAS HBsAg TEST DATE: 8/3/1999	ROCHE Enzymum-Test HBsAg III ES Systems TEST DATE: UNK	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 8/2/1999	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 6/7/1999	ABBOTT MUREX HBsAG Version 3 TEST DATE: UNK
9072-01	0.038	0.060	0.250	0.320	0.180	0.550	3	0.50
9072-02	0.019	0.060	0.180	0.290	0.210	0.640	1.360	0.50
9072-03	0.038	0.060	0.150	0.400	0.210	0.480	0.950	0.70
9072-04	0.058	0.040	0.180	0.530	0.130	0.510	0.250	0.50
9072-05	0.769	0.080	0.150	0.590	0.130	0.450	0.280	0.64
9072-06	0.038	0.140	0.100	0.340	0.210	0.530	0.140	0.54
9072-07	0.038	0.120	0.130	0.810	0.100	0.470	0.160	0.57
9072-08	0.038	0.160	0.130	0.390	0.180	0.480	0.560	0.57
9072-09	0.019	0.080	0.220	0.340	0.100	0.550	1.870	0.50
9072-10	0.038	0.040	0.200	0.540	0.100	0.680	1.290	0.50
9072-11	0.019	0.080	0.180	0.630	0.480	0.530	1.320	0.53
9072-12	0.923	0.160	1.480	1.070	1.100	0.660	1.160	2.70
9072-13	38.462	10.510	47.470	27.500	25.170	8.810	1.590	31.05
9072-14	38.462	53.030	MAX ABS	138.210	MAX ABS	MAX ABS	5.130	31.18
9072-15	38.462	55.840	MAX ABS	145.130	MAX ABS	MAX ABS	11.540	31.58
9072-16	38.462	56.250	MAX ABS	146.900	MAX ABS	MAX ABS	41.370	31.10
9072-17	38.462	55.170	MAX ABS	148.870	MAX ABS	MAX ABS	40.900	31.26
9072-18	35.577	54.000	MAX ABS	148.260	MAX ABS	MAX ABS	40.020	31.37

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



**HBV Seroconversion Panel Donor No.66481
Part Number: HBV9072**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ROCHE ELECSYS HBsAg TEST DATE: 6/15/1999	ROCHE Cobas Core HBsAg II EIA TEST DATE: 7/26/1999	ABBOTT CORZYME ANTI-HBc IgM(rDNA) TEST DATE: 6/8/1999	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	ABBOTT (OD) ANTI-HBc TOTAL TEST DATE: 5/27/1997	ABBOTT HBe Ag (OD) TEST DATE: 6/9/1999	SORIN ETI-MAK 3 TEST DATE: 6/17/1999
9072-01	0.438	0.160	0.038	Negative	0.584	0.369	0.170
9072-02	0.451	0.190	0.038	Negative	0.516	0.274	0.180
9072-03	0.455	0.200	0.038	Negative	0.480	0.250	0.140
9072-04	0.439	0.200	0.038	Negative	0.545	0.369	0.110
9072-05	0.46	0.210	0.038	Negative	0.490	0.536	0.120
9072-06	0.442	0.210	0.057	Negative	0.630	0.262	0.120
9072-07	0.472	0.220	0.038	Negative	0.407	0.298	0.120
9072-08	0.451	0.180	0.057	Negative	0.545	0.310	0.170
9072-09	0.482	0.210	0.038	Negative	0.500	0.250	0.170
9072-10	0.421	0.220	0.038	Negative	0.521	0.238	0.150
9072-11	0.439	0.190	0.057	Negative	0.545	0.286	0.080
9072-12	1.060	1.150	0.038	82000	0.564	0.869	0.530
9072-13	32.160	51.440	0.038	1280000	0.614	5.345	21.300
9072-14	687.400	72.470	0.038	Not Tested	0.655	20.000	MAX ABS
9072-15	2998.000	72.470	0.038	>110000000	0.407	22.262	MAX ABS
9072-16	2425.000	72.470	0.132	>110000000	0.952	>23.810	MAX ABS
9072-17	1755.000	72.470	0.289	>110000000	1.738	>23.810	MAX ABS
9072-18	600.000	72.470	0.906	>110000000	2.580	>23.810	MAX ABS

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



**HBV Seroconversion Panel Donor No.66481
Part Number: HBV9072**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT ANTI-HBc IgM (OD) TEST DATE: 6/8/1999	ORTHO VITROS ECi HBsAg TEST DATE: 5/18/2016	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 5/18/2016	ORTHO VITROS ECi HBeAg TEST DATE: 5/18/2016	ORTHO VITROS ECi ANTI-HBe TEST DATE: 5/18/2016	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 5/18/2016	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 5/18/2016
9072-01	0.038	0.05	3.21	0.36	0.15	0.01	1.2
9072-02	0.038	0.05	3.06	0.31	0.16	0.01	0.7
9072-03	0.038	0.05	3.16	0.3	0.16	0.01	0.00
9072-04	0.038	0.06	2.98	0.29	0.16	0.01	2.6
9072-05	0.038	0.05	3.14	0.28	0.16	0.01	1.3
9072-06	0.057	0.05	3.14	0.31	0.16	0.01	2.5
9072-07	0.38	0.05	3.14	0.31	0.06	0.01	2.3
9072-08	0.57	0.06	2.96	0.33	0.16	0.01	0.9
9072-09	0.038	0.05	2.97	0.31	0.16	0.01	2.3
9072-10	0.038	0.05	3.16	0.35	0.16	0.01	0.9
9072-11	0.057	Not Tested	Not Tested	Not Tested	Not Tested	Not Tested	Not Tested
9072-12	0.038	0.05	2.87	1.06	0.16	0.02	1.1
9072-13	0.038	0.06	3.07	19.3	0.09	0.01	1.8
9072-14	0.038	0.45	3.00	559	0.01	0.01	1.1
9072-15	0.038	5.39	2.88	2600	0.00	0.01	1.9
9072-16	0.132	33.5	1.28	3050	0.00	0.04	3.00
9072-17	0.287	40.4	0.72	2790	0.00	0.07	0.5
9072-18	0.906	45.3	0.25	2400	0.00	0.29	Not Tested

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com