



**HBV Seroconversion Panel Donor No.65099**  
**Part Number: HBV9073**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT HBV REAL TIME CP/ML	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	ABBOTT IMx HBsAg (V2) Reagent Pack	ABBOTT AxSYM HBsAg (V2) Reagent Pack
		TEST DATE : UNK	TEST DATE : UNK	TEST DATE : 7/26/1999	TEST DATE : 11/10/1999	TEST DATE : 6/23/1999	TEST DATE : 6/23/1999
9073-01	08/08/97	8	NOT DETECTED	0.350	0.339	0.480	0.430
9073-02	08/10/97	5	NOT DETECTED	0.310	0.403	0.460	0.440
9073-03	08/25/97	1	NOT DETECTED	0.380	0.194	0.470	0.430
9073-04	08/29/97	13	NOT DETECTED	0.350	0.242	0.920	0.400
9073-05	09/05/97	10	NOT DETECTED	0.280	0.177	0.440	0.420
9073-06	09/08/97	13	NOT DETECTED	0.310	0.194	0.470	0.410
9073-07	09/12/97	8	NOT DETECTED	0.310	0.758	0.460	0.460
9073-08	09/30/97	0	NOT DETECTED	0.350	0.355	0.470	0.450
9073-09	10/03/97	0	NOT DETECTED	0.310	0.339	0.500	0.430
9073-10	10/07/97	6	<52	0.350	0.210	0.450	0.410
9073-11	10/11/97	0	103	0.380	0.194	0.450	0.470
9073-12	10/14/97	1	304	0.100	0.226	0.720	0.440
9073-13	10/18/97	3	1129	0.310	0.355	0.520	0.470
9073-14	10/21/97	4	1918	0.980	0.403	0.60	0.590
9073-15	10/25/97	13	5659	3.350	1.113	1.270	1.080
9073-16	11/01/97	2	44953	28.360	7.161	7.750	5.490

PANEL MEMBER	ABBOTT PRISM HBsAg	BIOTEST ELISA HBsAg	ROCHE CAP-G/CTM HBV (cp/mL)	DADE BEHRING Enzygnost HBsAg 5.0	Abbott MUREX HBsAg Version 3	BIOMERIEUX VIDAS HBsAg	ROCHE Enzymum-Test HBsAg III ES Systems
	TEST DATE : UNK	TEST DATE : 6/22/1999	TEST DATE : 6/2/2005	TEST DATE : 6/7/1999	TEST DATE : UNK	TEST DATE : 8/3/1999	TEST DATE : UNK
9073-01	1.41	0.160	Not Tested	0.280	0.50	0.380	0.210
9073-02	0.85	0.190	Negative	0.180	0.51	0.500	0.130
9073-03	0.62	0.160	Not Tested	0.180	0.55	0.420	0.180
9073-04	0.35	0.160	Negative	0.180	0.56	0.030	0.100
9073-05	0.36	0.100	Negative	0.120	0.54	0.020	0.180
9073-06	0.34	0.060	Negative	0.130	0.51	0.010	0.210
9073-07	0.33	0.120	Negative	0.120	0.50	0.500	0.210
9073-08	0.93	0.040	Negative	0.130	0.50	0.540	0.080
9073-09	0.69	0.160	Negative	0.200	0.50	0.490	0.180
9073-10	0.46	0.140	Negative	0.170	0.49	0.440	0.180
9073-11	0.34	0.140	Negative	0.220	0.55	0.550	0.280
9073-12	0.66	0.160	257	0.250	0.57	0.420	0.230
9073-13	0.82	0.170	5130	0.350	0.81	0.760	0.310
9073-14	1.61	0.210	11600	0.750	1.12	0.720	0.460
9073-15	5.20	1.030	Not Tested	2.920	2.94	1.740	1.660
9073-16	36.58	7.680	223000	21.870	18.09	10.030	10.910

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HBV Seroconversion Panel Donor No.65099**  
**Part Number: HBV9073**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT ANTI-Hbe (OD) TEST DATE : 6/8/1999	ORTHO VITROS ECi HBsAg TEST DATE : 5/18/2016	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE : 5/18/2016	ORTHO VITROS ECi HBeAg TEST DATE : 5/18/2016	ORTHO VITROS ECi ANTI-HBe TEST DATE : 5/18/2016	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE : 5/18/2016	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE : 5/18/2016
9073-01	0.593	0.11	3.2	0.33	0.17	0.02	Not Tested
9073-02	0.630	0.12	3.47	0.4	0.05	0.02	Not Tested
9073-03	0.635	0.12	3.22	0.47	0.16	0.02	3.4
9073-04	0.613	0.13	3.53	0.53	0.15	0.03	0.8
9073-05	0.635	0.12	3.35	0.48	0.17	0.02	2.3
9073-06	0.593	0.12	3.31	0.4	0.16	0.02	1.4
9073-07	0.658	0.11	3.34	0.37	0.17	0.02	0.00
9073-08	0.624	0.11	3.35	0.36	0.16	0.02	1.9
9073-09	0.624	0.1	3.47	0.34	0.16	0.02	1.3
9073-10	0.635	0.12	3.54	0.35	0.16	0.02	1.7
9073-11	0.613	0.12	3.21	0.32	0.16	0.02	1.00
9073-12	0.652	0.13	3.2	0.34	0.16	0.02	0.6
9073-13	0.640	0.19	3.1	0.33	0.15	0.02	1.6
9073-14	0.584	0.46	3.08	0.33	0.15	0.02	1.8
9073-15	0.365	2.28	2.78	0.36	0.16	0.02	2.1
9073-16	0.521	21.9	3.12	0.66	0.15	0.02	0.1

	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE : 8/2/1999	ORTHO HBsAg ELISA Testsystem 3 TEST DATE : 6/7/1999	ROCHE Elecsys HBsAg TEST DATE : 6/15/1999	SORIN ETI-MAK 3 TEST DATE : 6/17/1999	ORTHO VITROS HBsAg Reagent Pack TEST DATE : UNK	ABBOTT ANTI-HBc TOTAL (OD) TEST DATE : 5/27/1999	ABBOTT HBeAg (OD) TEST DATE : 5/28/1999
9073-01	0.540	0.300	0.467	0.170	0.260	0.462	0.216
9073-02	0.530	0.180	0.489	0.170	0.300	0.415	0.216
9073-03	0.580	0.140	0.472	0.140	0.300	0.471	0.289
9073-04	0.630	0.120	0.466	0.120	0.310	0.495	0.289
9073-05	0.730	0.180	0.451	0.120	0.310	0.551	0.237
9073-06	0.460	0.160	0.436	0.110	0.290	0.522	0.247
9073-07	0.450	0.120	0.458	0.270	0.300	0.522	0.247
9073-08	0.530	0.160	0.461	0.180	0.260	0.445	0.258
9073-09	0.520	0.120	0.450	0.150	0.260	0.466	0.258
9073-10	0.810	0.180	0.480	0.190	0.270	0.545	0.443
9073-11	0.550	0.160	0.455	0.120	0.260	0.533	0.206
9073-12	0.620	0.160	0.489	0.120	0.310	0.533	0.237
9073-13	0.550	0.420	0.646	0.220	0.420	0.522	0.247
9073-14	0.620	0.790	0.875	0.400	0.780	0.482	0.268
9073-15	1.090	1.690	2.050	1.220	2.660	0.490	0.299
9073-16	4.980	13.550	13.140	10.260	22.800	0.458	0.412

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)