



**HBV Seroconversion Panel Donor No.66213  
Part Number: HBV9074**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)  TEST DATE: 4/26/1999	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL)  TEST DATE: UNK	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 7/26/1999	ABBOTT Auszyme Monoclonal HBsAg  TEST DATE: UNK	ABBOTT IMx HBsAg (V2) Reagent Pack  TEST DATE: 6/23/1999	ABBOTT AxSYM HBsAg (V2) Reagent Pack  TEST DATE: 6/23/1999
9074-01	12/07/97	10	NOT DETECTED	NEGATIVE	0.170	0.173	0.480	0.420
9074-02	12/14/97	6	NOT DETECTED	NEGATIVE	0.210	0.038	0.400	0.450
9074-03	12/28/97	4	NOT DETECTED	NOT TESTED	0.170	0.038	0.480	0.430
9074-04	12/31/97	13	NOT DETECTED	NEGATIVE	0.210	0.558	0.430	0.410
9074-05	01/04/98	6	NOT DETECTED	NEGATIVE	0.210	0.327	0.450	0.450
9074-06	01/07/98	5	NOT DETECTED	NEGATIVE	0.240	1.038	0.410	0.420
9074-07	01/11/98	9	NOT DETECTED	NEGATIVE	0.240	0.038	0.400	0.480
9074-08	01/14/98	6	NOT DETECTED	NEGATIVE	0.210	0.038	0.450	0.390
9074-09	01/18/98	6	NOT DETECTED	NEGATIVE	0.210	0.038	1.510	0.430
9074-10	01/21/98	4	NOT DETECTED	NEGATIVE	0.170	0.019	0.440	0.460
9074-11	01/25/98	6	NOT DETECTED	NEGATIVE	0.170	0.077	0.450	0.490
9074-12	01/28/98	8	<52	NEGATIVE	0.310	0.058	0.410	0.470
9074-13	02/01/98	14	NOT DETECTED	NOT TESTED	0.280	0.038	0.450	0.500
9074-14	02/04/98	9	66	NOT TESTED	0.350	0.058	0.440	0.410
9074-15	02/08/98	12	429	NOT TESTED	0.350	0.135	0.450	0.460
9074-16	02/11/98	8	939	NOT TESTED	0.420	0.250	0.610	0.560
9074-17	02/15/98	9	2902	NOT TESTED	1.220	0.577	0.820	0.560
9074-18	02/18/98	7	8969	NOT TESTED	2.970	1.212	2.22	0.790
9074-19	02/22/98	8	48168	107000	8.790	5.231	3.000	1.810
9074-20	02/25/98	7	74603	214000	21.210	9.615	6.380	3.190

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV9074 Rev03  
Effective Date: 09/22/2021



**HBV Seroconversion Panel Donor No.66213  
Part Number: HBV9074**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM HBsAg	BIOTEST ELISA HBsAg	DADE BEHRING Enzygnost HbsAg 5.0	ABBOTT MUREX HBsAg Version 3	ABBOTT CORZYME-M (r-DNA)	ROCHE COBAS CORE HBsAg II EIA	ORTHO VITROS HBsAg Reagent Pack	BIOMERIEUX VIDAS HBsAg
	TEST DATE: 6/21/1999	TEST DATE: 6/22/1999	TEST DATE: 6/7/1999	TEST DATE: 8/27/2009	TEST DATE: 6/8/1999	TEST DATE: 7/26/1999	TEST DATE: UNK	TEST DATE: 8/3/1999
9074-01	0.66	0.120	0.130	0.50	0.075	0.250	0.100	0.280
9074-02	0.95	0.140	0.120	0.49	0.066	0.200	0.100	0.440
9074-03	0.55	0.170	0.170	0.52	0.057	0.200	0.120	0.260
9074-04	0.58	0.140	0.150	0.45	0.075	0.210	0.110	0.260
9074-05	0.63	0.100	0.280	0.50	0.075	0.190	0.100	0.290
9074-06	0.63	0.120	0.170	0.53	0.075	0.190	0.100	0.350
9074-07	0.59	0.100	0.170	0.47	0.057	0.180	0.100	0.260
9074-08	0.61	0.120	0.180	0.46	0.075	0.190	0.100	0.350
9074-09	0.29	0.040	0.100	0.45	0.057	0.190	0.090	0.470
9074-10	0.75	0.170	0.120	0.47	0.057	0.200	0.090	0.720
9074-11	0.56	0.170	0.130	0.47	0.075	0.210	0.100	0.420
9074-12	0.73	0.120	0.120	0.46	0.075	0.160	0.100	0.470
9074-13	0.68	0.160	0.150	0.51	0.075	0.180	0.100	0.470
9074-14	0.69	0.060	0.150	0.57	0.057	0.190	0.110	0.620
9074-15	0.35	0.120	0.230	0.60	0.057	0.250	0.110	0.470
9074-16	0.58	0.160	0.480	0.67	0.094	0.360	0.170	0.530
9074-17	2.12	0.390	0.930	1.17	0.038	0.700	0.510	0.720
9074-18	4.38	0.810	2.420	2.21	0.075	1.530	1.910	1.780
9074-19	14.99	2.770	9.280	7.21	0.057	5.240	8.220	4.430
9074-20	23.64	3.780	21.850	14.01	0.057	10.440	18.500	8.2630

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV9074 Rev03  
Effective Date: 09/22/2021



**HBV Seroconversion Panel Donor No.66213  
Part Number: HBV9074**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ROCHE ENZYMUM-TEST HBsAg III ES SYSTEMS  TEST DATE: UNK	ORGANON TEKNIKA HEPANOSTIKA HBsAg UNI-FORM II  TEST DATE: 8/3/1999	ORTHO HBsAg ELISA TEST SYSTEM 3  TEST DATE: 7/13/1999	ROCHE ELECSYS HBsAg  TEST DATE: 6/15/1999	ORTHO VITROS ECi HBsAg  TEST DATE: 8/30/2018	SORIN ETI-MAK 3  TEST DATE: 6/17/1999	ABBOTT (OD)  TEST DATE: 6/10/1999	ORTHO VITROS ECi HBeAg  TEST DATE: 8/30/2018
9074-01	0.210	0.540	0.160	0.560	0.06	0.180	0.250	0.25
9074-02	0.180	0.640	0.110	0.480	0.07	0.150	0.238	0.20
9074-03	0.180	0.400	0.070	0.450	0.07	0.170	0.333	0.20
9074-04	0.150	0.0590	0.110	0.500	0.07	0.170	0.214	0.21
9074-05	0.210	0.580	0.040	0.460	0.05	0.170	0.667	0.20
9074-06	0.130	0.480	0.070	0.420	0.39	0.170	0.262	0.21
9074-07	0.030	0.470	0.070	0.430	0.05	0.110	0.238	0.20
9074-08	0.260	0.400	0.070	0.410	0.05	0.090	0.262	0.20
9074-09	0.100	0.470	0.050	0.430	0.05	0.120	0.369	0.20
9074-10	0.260	0.550	0.050	0.460	0.05	0.110	0.310	0.20
9074-11	0.310	0.550	0.160	0.500	0.05	0.180	0.286	0.19
9074-12	0.230	0.520	1.370	0.450	0.04	0.170	0.250	0.20
9074-13	0.210	0.500	0.070	0.450	0.06	0.180	0.357	0.20
9074-14	0.260	0.430	0.090	0.440	0.06	0.180	0.298	0.19
9074-15	0.180	0.600	0.110	0.530	0.14	0.150	0.250	0.18
9074-16	0.260	0.600	0.110	0.530	0.33	0.270	0.286	0.18
9074-17	0.460	0.570	0.460	0.860	0.26	0.450	0.429	0.21
9074-18	1.050	0.840	1.220	1.720	1.60	0.970	0.583	0.23
9074-19	3.580	2.090	4.420	4.970	5.01	2.980	0.595	0.31
9074-20	10.830	3.730	11.980	12.410	12.7	6.630	1.143	0.70

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV9074 Rev03  
Effective Date: 09/22/2021



**HBV Seroconversion Panel Donor No.66213  
Part Number: HBV9074**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi ANTI-HBs QUANT	ABBOTT ANTI-HBc TOTAL (OD)	ABBOTT ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc IgM	ABBOTT ANTI-HBe (OD)	ORTHO VITROS ECi ANTI-HBe
	TEST DATE: 8/30/2018	TEST DATE: 6/18/1999	TEST DATE: 6/8/1999	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 6/8/1999	TEST DATE: 8/30/2018
9074-01	0.0	0.910	0.076	2.94	0.01	0.664	0.15
9074-02	0.5	0.690	0.067	3.02	0.02	0.646	0.17
9074-03	0.0	0.784	0.057	3.01	0.02	0.619	0.19
9074-04	0.0	0.740	0.076	2.98	0.02	0.652	0.18
9074-05	0.4	0.817	0.076	2.96	0.02	0.664	0.18
9074-06	0.5	0.770	0.076	2.80	0.02	0.652	0.19
9074-07	0.3	0.690	0.057	2.84	0.02	0.646	0.19
9074-08	0.9	0.741	0.076	3.04	0.02	0.664	0.19
9074-09	0.2	0.817	0.057	3.06	0.02	0.640	0.20
9074-10	0.0	0.754	0.057	2.77	0.02	0.603	0.22
9074-11	0.5	0.615	0.076	3.04	0.01	0.629	0.18
9074-12	0.3	0.678	0.076	2.92	0.02	0.664	0.17
9074-13	0.0	0.833	0.076	3.00	0.01	0.670	0.19
9074-14	0.0	0.690	0.057	2.91	0.02	0.629	0.19
9074-15	0.0	0.784	0.057	2.86	0.02	0.646	0.20
9074-16	0.1	0.645	0.094	2.91	0.02	0.635	0.18
9074-17	0.7	0.645	0.038	3.04	0.01	0.658	0.17
9074-18	1.4	0.645	0.076	3.00	0.02	0.670	0.20
9074-19	1.5	0.728	0.057	2.86	0.02	0.613	0.18
9074-20	0.0	0.701	0.057	2.83	0.02	0.629	0.18

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)