

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 69089 Part Number: HCV10000

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/22/2000	TEST DATE: 7/17/2000
10000-01	09/23/98	5	NEGATIVE	0.590	0.486	0.686	0.210	0.290
10000-02	10/01/98	6	NEGATIVE	0.343	0.383	0.578	0.170	0.210
10000-03	10/06/98	6	NEGATIVE	0.313	0.372	0.756	0.190	0.200
10000-04	10/24/98	24	POSITIVE	0.345	0.497	0.733	0.160	0.250
10000-05	10/29/98	40	POSITIVE	0.342	0.679	0.689	0.140	0.190
10000-06	11/01/98	24	POSITIVE	0.331	1.003	0.689	0.150	0.200
10000-07	11/05/98	29	POSITIVE	0.318	0.955	0.622	0.130	0.190
10000-08	11/08/98	38	POSITIVE	0.318	0.617	0.689	0.140	0.220
10000-09	11/12/98	108	POSITIVE	0.411	1.479	0.556	0.130	0.180
10000-10	11/15/98	216	POSITIVE	0.475	0.724	0.600	0.300	0.530

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO VITROS ANTI-HCV Reagent Pack Lab # 5	ORTHO Enhanced SAVE ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/22/2000	TEST DATE: 7/25/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10000-01	0.260	1.205	0.210	0.00	0.05	0.13	0.10
10000-02	0.230	1.240	0.170	0.00	0.06	0.08	0.11
10000-03	0.180	1.039	0.190	0.00	0.06	0.07	0.10
10000-04	0.190	1.006	0.160	0.00	0.05	0.06	0.10
10000-05	0.170	1.053	0.140	0.00	0.05	0.05	0.10
10000-06	0.180	1.151	0.150	0.00	0.06	0.07	0.09
10000-07	0.170	1.119	0.130	0.00	0.06	0.05	0.12
10000-08	0.230	1.041	0.140	0.00	0.05	0.06	0.10
10000-09	0.280	1.299	0.130	0.00	0.04	0.05	0.10
10000-10	0.610	1.310	0.300	0.020	0.05	0.19	0.10

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10000 Rev03  
Effective Date: 10/11/2021