



HCV Seroconversion Panel Donor No. 69158
Part Number: HCV10003

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/22/2000	TEST DATE: 7/17/2000
10003-01	09/28/98	10	NEGATIVE	0.097	0.321	0.156	0.140	0.090
10003-02	09/30/98	6	NEGATIVE	0.092	0.314	0.133	0.120	0.150
10003-03	10/14/98	12	NEGATIVE	0.098	0.317	0.133	0.130	0.250
10003-04	10/17/98	7	NEGATIVE	0.103	0.317	0.111	0.140	0.190
10003-05	10/21/98	13	NEGATIVE	0.097	0.338	0.089	0.170	0.180
10003-06	10/24/98	9	POSITIVE	0.107	0.420	0.111	0.100	0.130
10003-07	10/28/98	11	POSITIVE	0.100	4.311	0.111	0.140	0.140
10003-08	11/02/98	26	POSITIVE	0.103	2.730	0.111	0.130	0.140
10003-09	11/04/98	25	POSITIVE	0.101	2.863	0.089	0.120	0.110
10003-10	11/09/98	80	POSITIVE	0.097	2.352	0.089	0.110	0.130
10003-11	11/11/98	128	POSITIVE	0.127	1.519	0.111	0.120	0.110
10003-12	11/16/98	284	POSITIVE	0.137	1.901	0.089	0.100	0.140
10003-13	11/18/98	140	POSITIVE	0.128	2.235	0.089	0.110	0.100

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE Cobas Core ANTI-HCV EIA Lab # 5	ORTHO Vitros ANTI-HCV Reagent Pack Lab # 5	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV 1/2
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/6/2000	TEST DATE: 8/22/2000	TEST DATE: 7/25/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10003-01	0.080	0.125	0.070	0.140	0.000	0.05	0.01	0.09
10003-02	0.250	0.279	0.040	0.120	0.000	0.06	0.01	0.08
10003-03	1.450	1.243	0.040	0.130	0.000	0.05	0.01	0.08
10003-04	1.600	1.288	0.040	0.140	0.000	0.06	0.01	0.08
10003-05	1.000	1.111	0.040	0.170	0.000	0.04	0.01	0.09
10003-06	0.830	0.937	0.040	0.100	0.000	0.04	0.01	0.10
10003-07	0.810	0.944	0.040	0.140	0.000	0.05	0.01	0.09
10003-08	0.670	0.799	0.040	0.130	0.000	0.05	0.01	0.09
10003-09	0.640	0.734	0.060	0.120	0.000	0.05	0.01	0.09
10003-10	0.530	0.652	0.050	0.110	0.000	0.05	0.01	0.09
10003-11	0.460	0.611	0.060	0.120	0.000	0.05	0.01	0.09
10003-12	0.470	0.611	0.060	0.100	0.000	0.06	0.01	0.09
10003-13	0.330	0.515	0.050	0.110	0.000	0.04	0.01	0.10

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10003 Rev03
 Effective Date: 10/11/2021