



**HCV Seroconversion Panel Donor No. 67355  
Part Number: HCV10007**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT PRISM HCV Lab # 5	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/4/2000	TEST DATE: 7/20/2000
10007-01	04/14/98	6	NEGATIVE	0.141	0.324	0.026	0.060	0.130
10007-02	04/16/98	7	NEGATIVE	0.101	0.300	0.158	0.070	0.090
10007-03	04/21/98	4	NEGATIVE	0.104	0.300	0.289	0.060	0.090
10007-04	04/23/98	13	NEGATIVE	0.089	0.293	0.211	0.060	0.150
10007-05	04/28/98	10	NEGATIVE	0.112	0.283	0.289	0.070	0.110
10007-06	04/30/98	6	NEGATIVE	0.104	0.310	0.289	0.060	0.110
10007-07	05/07/98	NOT TESTED	NOT TESTED	0.144	0.434	0.237	0.080	0.110
10007-08	05/09/98	9	POSITIVE	0.189	1.676	0.263	0.060	0.100
10007-09	05/14/98	4	POSITIVE	0.227	0.448	0.289	0.070	0.140
10007-10	05/19/98	9	POSITIVE	0.150	0.393	0.237	0.070	0.100
10007-11	05/21/98	7	POSITIVE	0.129	0.366	0.316	0.070	0.090

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE Cobas Core ANTI-HCV EIA Lab # 5	ORTHO Enhanced SAVe ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/6/2000	TEST DATE: 9/18/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10007-01	0.080	0.134	0.090	0.000	0.06	0.03	0.14
10007-02	0.050	0.100	0.070	0.000	0.05	0.01	0.11
10007-03	0.070	0.125	0.080	0.000	0.06	0.02	0.13
10007-04	0.040	0.095	0.070	0.000	0.06	0.01	0.12
10007-05	0.080	0.109	0.070	0.000	0.06	0.02	0.12
10007-06	0.070	0.119	0.070	0.000	0.27	0.02	0.13
10007-07	0.110	0.148	0.080	0.000	0.49	0.05	0.17
10007-08	0.070	0.120	0.080	0.000	0.05	0.02	0.12
10007-09	0.090	0.123	0.090	0.000	0.05	0.03	0.12
10007-10	0.060	0.108	0.070	0.000	0.06	0.02	0.12
10007-11	0.050	0.118	0.080	0.010	0.05	0.01	0.12

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10007 Rev03  
Effective Date: 10/11/2021