



**HCV Seroconversion Panel Donor No. 69263  
Part Number: HCV10008**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ALPHA	ABBOTT	ABBOTT	ABBOTT	ABBOTT	ABBOTT
		Normal Range (10-45 u/L)	HCV by PCR	MUREX ANTI-HCV 4.0	MUREX HCV Ag/Ab Combo	ANTI-HCV Lab # 1	ANTI-HCV 3.0 Lab # 5	PRISM HCV Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/21/2000	TEST DATE: 10/4/2000
10008-01	10/27/98	5	NEGATIVE	0.097	0.286	0.111	0.080	0.070
10008-02	10/29/98	5	NEGATIVE	0.086	0.310	0.089	0.070	0.060
10008-03	11/04/98	9	NEGATIVE	0.095	0.328	0.111	0.080	0.060
10008-04	11/06/98	8	POSITIVE	0.092	0.324	0.111	0.070	0.060
10008-05	11/11/98	6	POSITIVE	0.086	7.279	0.111	0.070	0.060
10008-06	11/13/98	11	POSITIVE	0.095	5.552	0.111	0.070	0.060
10008-07	11/18/98	14	POSITIVE	0.094	9.800	0.111	0.070	0.050
10008-08	11/20/98	12	POSITIVE	0.083	9.831	0.111	0.070	0.060
10008-09	11/25/98	22	POSITIVE	0.091	11.121	0.111	0.070	0.070
10008-10	11/27/98	19	POSITIVE	0.086	11.248	0.089	0.060	0.060

PANEL MEMBER	ABBOTT	ABBOTT	ABBOTT	ROCHE	ORTHO	ORTHO	ORTHO	ORTHO	
	AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5	Architect ANTI-HCV	PRISM ANTI-HCV	Cobas Core ANTI-HCV EIA Lab # 5	Enhanced SAvE ANTI-HCV 3.0 Lab # 5	ECi HBsAg	ECi ANTI-HCV	ECi ANTI-HIV 1/2	
		TEST DATE: 7/20/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/6/2000	TEST DATE: 9/18/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10008-01	0.110	0.100	0.135	0.060	0.000	0.06	0.01	0.12	
10008-02	0.080	0.090	0.095	0.050	0.000	0.05	0.00	0.11	
10008-03	0.080	0.200	0.112	0.060	0.000	0.05	0.01	0.12	
10008-04	0.090	0.090	0.114	0.060	0.000	0.04	0.01	0.12	
10008-05	0.090	0.060	0.105	0.060	0.000	0.04	0.00	0.13	
10008-06	0.170	0.090	0.121	0.050	0.000	0.05	0.01	0.11	
10008-07	0.140	0.090	0.118	0.060	0.010	0.05	0.01	0.10	
10008-08	0.110	0.080	0.119	0.060	0.000	0.04	0.01	0.12	
10008-09	0.090	0.100	0.127	0.060	0.000	0.05	0.01	0.11	
10008-10	0.110	0.080	0.117	0.060	0.000	0.04	0.01	0.11	

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10008 Rev03  
Effective Date: 10/11/2021