

# ZeptoMetrix®

## HCV Seroconversion Panel Donor No. 68896 Part Number: HCV10009

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

### HCV Genotype: 1a/1b

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ALPHA HCV by PCR TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 9/2/1999	ABBOTT ANTI-HCV 3.0 Lab # 5 TEST DATE: 9/21/2000	ABBOTT PRISM HCV Lab # 5 TEST DATE: 10/4/2000
10009-01	09/23/98	11	NEGATIVE	0.297	0.313	0.200	0.130	0.090
10009-02	09/25/98	23	NEGATIVE	0.263	0.330	0.178	0.110	0.090
10009-03	10/01/98	47	NEGATIVE	0.201	0.337	0.156	0.090	0.110
10009-04	10/03/98	44	POSITIVE	0.224	0.367	0.133	0.110	0.080
10009-05	10/08/98	34	POSITIVE	0.215	0.398	0.178	0.090	0.080
10009-06	10/10/98	23	POSITIVE	0.188	0.354	0.156	0.110	0.080

PANEL MEMBER	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5 TEST DATE: 7/20/2000	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ROCHE Cobas Core ANTI-HCV EIA TEST DATE: 9/6/2000	ORTHO Enhanced SAVE ANTI-HCV 3.0 Lab # 5 TEST DATE: 9/18/2000	ORTHO Eci HBsAg TEST DATE: 11/16/2018	ORTHO Eci ANTI-HCV TEST DATE: 11/16/2018	ORTHO Eci ANTI-HIV ½ TEST DATE: 11/16/2018
10009-01	0.170	0.080	0.120	0.270	0.000	0.07	0.02	0.12
10009-02	0.170	0.060	0.108	0.150	0.000	0.07	0.01	0.12
10009-03	0.110	0.050	0.102	0.110	0.000	0.05	0.01	0.13
10009-04	0.150	0.050	0.098	0.100	0.000	0.05	0.01	0.11
10009-05	0.160	0.060	0.116	0.090	0.000	0.06	0.01	0.12
10009-06	0.150	0.050	0.134	0.100	0.000	0.05	0.01	0.14

#### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10009 Rev04  
Effective Date: 10/11/2021