



**HCV Seroconversion Panel Donor No. 68628
Part Number: HCV10016**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 2

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: 9/19/2000
10016-01	08/02/98	10	NEGATIVE	0.101	0.293	0.158	0.090
10016-02	08/04/98	15	NEGATIVE	0.104	0.296	0.184	0.100
10016-03	08/10/98	5	NEGATIVE	0.111	0.286	0.184	0.140
10016-04	08/14/98	9	NEGATIVE	0.104	0.279	0.184	0.140
10016-05	08/28/98	11	NEGATIVE	0.117	0.293	0.211	0.100
10016-06	09/01/98	10	NEGATIVE	0.117	0.293	0.184	0.140
10016-07	09/05/98	9	NEGATIVE	0.134	0.313	0.184	0.100
10016-08	09/09/98	4	NEGATIVE	0.107	0.316	0.158	0.170
10016-09	09/14/98	6	POSITIVE	0.108	0.289	0.184	0.110
10016-10	09/18/98	7	POSITIVE	0.108	8.752	0.184	0.090
10016-11	10/09/98	15	POSITIVE	0.125	8.704	0.211	0.120
10016-12	10/12/98	41	POSITIVE	0.113	8.000	0.211	0.230

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/19/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10016-01	0.050	0.090	0.000	0.06	0.01	0.14
10016-02	0.050	0.090	0.000	0.06	0.01	0.13
10016-03	0.060	0.118	0.000	0.07	0.02	0.13
10016-04	0.060	0.147	0.000	0.06	0.01	0.13
10016-05	0.060	0.096	0.000	0.07	0.02	0.15
10016-06	0.070	0.116	0.000	0.06	0.02	0.16
10016-07	0.070	0.104	0.010	0.06	0.04	0.13
10016-08	0.050	0.096	0.000	0.06	0.02	0.17
10016-09	0.070	0.129	0.010	0.06	0.03	0.14
10016-10	0.060	0.116	0.010	0.06	0.02	0.13
10016-11	0.080	0.146	0.020	0.08	0.06	0.31
10016-12	0.090	0.562	0.010	0.09	0.06	0.26

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10016 Rev04
Effective Date: 10/12/2021