



**HCV Seroconversion Panel Donor No. 69142
Part Number: HCV10019**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4 0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/27/1999	TEST DATE: 9/19/2000
10019-01	09/26/98	18	NEGATIVE	0.235	0.346	0.230	0.210
10019-02	10/04/98	13	NEGATIVE	0.218	0.356	0.246	0.290
10019-03	10/19/98	7	NEGATIVE	0.205	0.383	0.213	0.260
10019-04	10/21/98	8	NEGATIVE	0.177	0.403	0.230	0.220
10019-05	10/26/98	10	NEGATIVE	0.186	0.383	0.213	0.260
10019-06	10/29/98	11	POSITIVE	0.180	1.447/0.936	0.230	0.270
10019-07	11/02/98	48	POSITIVE	0.182	0.488	0.230	0.140
10019-08	11/07/98	38	POSITIVE	0.176	0.468	0.230	0.170

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/19/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10019-01	0.100	0.150	0.020	0.09	0.02	0.13
10019-02	0.100	0.119	0.010	0.08	0.04	0.13
10019-03	0.100	0.119	0.010	0.12	0.04	0.12
10019-04	0.100	0.090	0.010	0.10	0.03	0.13
10019-05	0.090	0.098	0.010	0.11	0.03	0.13
10019-06	0.080	0.102	0.010	0.09	0.03	0.12
10019-07	0.080	0.091	0.010	0.10	0.03	0.11
10019-08	0.070	0.098	0.010	0.08	0.03	0.12

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com