



**HCV Seroconversion Panel Donor No. 66801  
Part Number: HCV10025**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1a**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4 0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT AxSYM ANTI-HCV3.0 Reagent Pack Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: UNK
10025-01	02/19/98	11	NEGATIVE	0.123	0.303	0.135	0.170
10025-02	02/24/98	9	NEGATIVE	0.150	0.303	0.162	0.200
10025-03	02/26/98	11	NEGATIVE	0.123	0.286	0.162	0.150
10025-04	03/03/98	8	NEGATIVE	0.119	0.354	0.081	0.240
10025-05	03/05/98	8	NEGATIVE	0.104	0.327	0.135	0.180
10025-06	03/12/98	10	NEGATIVE	0.119	0.337	0.162	0.140
10025-07	03/16/98	7	NEGATIVE	0.120	0.316	0.135	0.170
10025-08	03/19/98	7	POSITIVE	0.120	0.871	0.135	0.170
10025-09	03/31/98	23	POSITIVE	0.131	10.350	0.054	0.210
10025-10	04/13/98	51	POSITIVE	0.126	9.541	0.162	0.280
10025-11	04/16/98	132	POSITIVE	0.162	6.612	0.162	0.290

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT Prism ANTI-HCV	SANOFI Monolisa ANTI-HCV PLUS Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV 1/2
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/20/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10025-01	0.060	0.120	0.200	0.07	0.03	0.10
10025-02	0.080	0.102	0.180	0.08	0.05	0.09
10025-03	0.060	0.129	0.180	0.07	0.02	0.09
10025-04	0.070	0.118	0.200	0.07	0.02	0.09
10025-05	0.050	0.154	0.180	0.07	0.02	0.09
10025-06	0.060	0.124	0.180	0.07	0.02	0.09
10025-07	0.050	0.119	0.180	0.06	0.02	0.09
10025-08	0.060	0.118	0.190	0.07	0.02	0.09
10025-09	0.070	0.171	0.210	0.07	0.03	0.10
10025-10	0.060	0.149	0.180	0.07	0.03	0.10
10025-11	0.150	0.175	0.220	0.06	0.05	0.09

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10025 Rev04  
Effective Date: 10/13/2021