

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 67116 Part Number: HCV10028

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 2

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/25/1999
10028-01	03/17/98	14	NEGATIVE	0.152	0.322	0.135
10028-02	03/19/98	9	NEGATIVE	0.142	0.325	0.108
10028-03	03/24/98	9	NEGATIVE	0.152	0.332	0.108
10028-04	03/26/98	8	NEGATIVE	0.148	0.394	0.135
10028-05	04/16/98	7	POSITIVE	0.167	1.092	0.216
10028-06	04/21/98	196	POSITIVE	0.151	1.305	0.162
10028-07	04/23/98	352	POSITIVE	0.158	0.575	0.162

PANEL MEMBER	ABBOTT ANTI-HCV 3.0 Lab # 5	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO VITROS ECi ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5
	TEST DATE: 10/16/2000	TEST DATE: 10/17/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/29/2000
10028-01	0.070	0.130	0.040	0.370	0.06	0.010
10028-02	0.140	0.140	0.040	0.375	0.06	0.010
10028-03	0.150	0.140	0.040	0.458	0.07	0.020
10028-04	0.070	0.120	0.030	0.378	0.05	0.010
10028-05	0.130	0.160	0.050	0.400	0.08	0.020
10028-06	0.120	0.150	0.040	0.472	0.07	0.010
10028-07	0.390	0.230	0.050	0.384	0.07	0.040

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10028 Rev04
Effective Date: 10/13/2021