



**HCV Seroconversion Panel Donor No.68629  
Part Number: HCV10033**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1b**

PANEL MEMBER	DATE OF DRAW	ALT	ALPHA	ABBOTT	ABBOTT	ABBOTT	ABBOTT
		Normal Range (10-45 u/L)	HCV by PCR	MUREX ANTI-HCV 4.0	MUREX HCV Ag/Ab Combo	ANTI-HCV Lab # 1	Architect ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/27/1999	TEST DATE: UNK
10033-01	08/24/98	8	NEGATIVE	0.105	0.325	0.197	0.100
10033-02	08/26/98	10	NEGATIVE	0.107	0.346	0.213	0.080
10033-03	09/01/98	9	NEGATIVE	0.101	0.329	0.230	0.070
10033-04	09/04/98	12	NEGATIVE	0.098	0.329	0.213	0.080
10033-05	09/09/98	8	NEGATIVE	0.105	0.318	0.213	0.080
10033-06	09/11/98	8	NEGATIVE	0.101	0.339	0.180	0.070
10033-07	09/16/98	10	NEGATIVE	0.095	0.325	0.180	0.070
10033-08	09/18/98	12	POSITIVE	0.092	0.332	0.197	0.070
10033-09	09/23/98	10	POSITIVE	0.086	3.849	0.164	0.070
10033-10	09/25/98	13	POSITIVE	0.087	0.459	0.213	0.070

PANEL MEMBER	ABBOTT PRISM ANTI-HCV	ORTHO HCV 3.0 ELISA Test system Enhanced SAVe	ROCHE Cobas Core ANTI-HCV EIA	ORTHO ECI HBsAg	ORTHO ECI ANTI-HCV	ORTHO ECI ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: 10/23/2000	TEST DATE: 9/7/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10033-01	0.129	0.000	0.200	0.11	0.02	0.08
10033-02	0.139	-0.010	0.200	0.16	0.02	0.09
10033-03	0.134	0.010	0.200	0.17	0.02	0.09
10033-04	0.163	-0.020	0.100	0.19	0.02	0.09
10033-05	0.196	0.000	0.100	0.18	0.01	0.10
10033-06	0.196	0.010	0.200	0.16	0.01	0.10
10033-07	0.327	0.000	0.100	0.14	0.01	0.10
10033-08	0.124	0.000	0.200	0.13	0.01	0.25
10033-09	0.122	0.000	0.100	0.07	0.01	0.09
10033-10	0.105	0.000	0.100	0.07	0.01	0.09

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)